
MTI Manual

**Memory Transfer Interface Instructions
for Prentke Romich Company
Communication Devices**

PRENTKE ROMICH COMPANY

1022 Heyl Rd. • Wooster, Ohio 44691

Telephone: (330) 262-1984

Fax: (330) 263-4829

E-Mail Address: info@prentrom.com

Web Site Address: www.prentrom.com

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
Prentke Romich International

Minerva House, Minerva Business Park
Lynch Wood, Peterborough, Cambridgeshire

PE2, 6FT UK

Fax: +44 1733 391939

E-Mail Address: info@prentromint.com



AlphaTalker, DeltaTalker, Liberator II, Vanguard, SideKick, AXS1600, Pathfinder, Vantage, SpringBoard, MTI and Memory Transfer Interface are trademarks of Prentke Romich Company.

MTI+ ©1999 Rehabmodul AB, Sweden

Minspeak is a registered trademark of Semantic Compaction Systems. Minspeak uses the methodology derived from the concepts covered by U.S Patent Numbers 4,661,916; 5,097,425; 5,210,689 and subsequent patents and applications of Bruce Baker.

IBM and PC-DOS are registered trademarks of International Business Machines, Inc.

Macintosh and AppleTalk are registered trademarks of Apple Computer, Inc.

Windows and MS-DOS are registered trademarks of Microsoft, Inc.

PaintShop Pro Version Shareware ©1990-1995 JASC, Inc. All rights reserved. Portions ©1994 Digital Workshop. Portions ©1991-1994 Access Softek.



Attention! Consult accompanying documents prior to use.

When this product is operated in a medical environment, do not use with any product that is not medically approved.

A Note about Cleaning PRC Products

Prentke Romich products can be cleaned with a warm, damp (not wet) cloth.

To disinfect, mix $\frac{1}{4}$ cup vinegar with 1 cup water.
Wipe product with this mixture and allow product to air dry.

©2003, 2004 Prentke Romich Company

All rights reserved. Under copyright laws this manual may not be copied, in whole or in part, without the written consent of Prentke Romich Company.

Table of Contents

Pathfinder

Transfer Tips	6
IBM Windows	9
DOS	11
Macintosh Tips	13
Macintosh	15
Transfer Vocabulary from One Pathfinder to Another	17
Importing Graphics	19
Transferring Notebooks	23
Importing Sound (wav) Files	24
Secure Digital Card or USB Connector Transfers	25

Vantage/Vanguard II (including Vantage Plus and Vanguard II Plus)

Tips	29
IBM Windows	31
DOS	33
Macintosh Tips	35
Macintosh	37
Transfer Vocabulary from One Device to Another	39
Transferring Notebooks	40
Importing Graphics	41
Importing Sound (wav) Files	45
Secure Digital Card or USB Connector Memory Transfers	46

SpringBoard

Transfer Tips	49
IBM Windows	51
DOS	53
Macintosh Tips	55
Macintosh	57
Transfer Vocabulary from One SpringBoard to Another	59
Importing Graphics	61
Importing Sound (wav) Files	65
Secure Digital Card or USB Connector Memory Transfers	66

Vanguard and Vanguard for Windows

Transfer Tips	69
IBM Windows	71
DOS	73
Macintosh Tips	75
Macintosh	77
Transfer Vocabulary from One Vanguard to Another	79
Import Graphics	81

AlphaTalker

Transfer Tips	87
IBM Windows.....	89
DOS	91
Macintosh Tips	93
Macintosh	95
Transfer Vocabulary from One AlphaTalker to Another.....	97
Keyboard Emulation	98
Tips for a Successful Emulation Transfer	99
Keyboard Emulation -IBM.....	100
Keyboard emulation – Macintosh.....	101
Copy the ASSIGN KEYBOARD EMULATION Key to your Overlay	102

DeltaTalker and DeltaTalker IR

Transfer Tips	105
IBM Windows.....	107
DOS	109
Macintosh Tips	111
Macintosh	113
Transfer Vocabulary from One DeltaTalker to Another.....	115

Appendices

Appendix A: Computer Key Commands for Keyboard Emulation.....	118
Appendix B: Troubleshooting.....	120

**MTI for the
Pathfinder**



Tips for a Successful Windows or DOS Memory Transfer



Plug in the battery charger before you begin a memory transfer.



DO NOT attempt a memory transfer by using the infrared capability. Use the cable that came with your MTI disks and connect your Pathfinder and computer with the cable.



Have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.



Before you begin a transfer with this MTI, remove any older versions of the MTI that you may have on your computer



Once the transfer is in progress, do not plug anything into, or unplug anything from, the Pathfinder.



When a transfer is in progress, periodically check the computer screen and Pathfinder display for any messages or problems.



If you experience a problem, try changing the baud rate to 57600 in the MTI menu bar. Also, double check your port setting.



You may want to make a backup copy of each individual User Area as well as a backup copy of the entire device. This will prove helpful if you experience technical difficulties with your Pathfinder at some other time.



Make sure **AppleTalk** and **SerialKeys** are turned **Off**.

☞ For All Pathfinder MTI Transfers

☞ Vocabulary, Activities and Notebooks are *specific* to each User Area. That is, they can only be accessed from a specific User Area.

☞ Imported icons, graphics and WAV files, the Speech Dictionary, the Word Prediction Dictionary, and IR commands are *universal* to all User Areas. That is, they can be accessed from any User Area.

☞ When you save a single User Area, the only imported graphics/icons/WAV files that will be saved are those that are used in that specific User Area. When you save the entire device contents, all imported graphics/icons will be saved.

☞ About Merging Vocabulary:

When you load vocabulary from one User Area into the Pathfinder you are asked if you want to **Erase Old Vocabulary**, **Merge: Keep Old if Conflict**, or **Merge: Replace Old if Conflict**. If you do not want to erase old vocabulary, you must select one of the Merge options. If the Pathfinder encounters conflicts between old and new vocabulary, you can tell it to **Keep the old vocabulary** (and ignore the new) or to **Replace the old vocabulary with the new vocabulary**.

☞ The Speech dictionary and any imported icons and graphics will be merged according to your directions. The Word Prediction Dictionary and any IR commands **will not** be merged. You will have to re-teach the IR commands to the Pathfinder.

☞ You cannot save the Word Prediction Dictionary with a single User Area or load it into a single User Area. If you have modified your Word Prediction Dictionary, you should use **Save Entire Device Contents** if you want a backup of the Dictionary. If you save the entire device contents, you cannot load just one User Area from that file back into the Pathfinder. You have to load the whole backup.

☞ When you **Erase Vocabulary** from the MAINTENANCE menu, the Speech and Word Prediction Dictionaries return to their default settings. Imported icons and graphics and any IR commands will be deleted.

About Baud Rates

When the Pathfinder and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the Pathfinder must be the same.**

The default baud rate for the Pathfinder is 57600. You can change the MTI baud rate to match the Pathfinder or you change the Pathfinder's baud rate to match the MTI program.

To Change the MTI Baud Rate to Match the Pathfinder:

If you are using DOS, change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>.

For Windows, change the baud rate in the MTI "Options" menu.

To Change the Pathfinder's Baud Rate to Match the MTI:

1. Go to the Toolbox.
2. Activate the TRANSFER MEMORY MENU.
3. Select "More Items".
4. Activate the "MTI Baud Rate" box until you see the baud rate you want to use.
5. Activate OK.

Save Vocabulary on an IBM Computer Using the MTI for Windows[®]

➡ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you begin.

To save memory to an IBM computer running Windows or to load memory back into the Pathfinder:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your Pathfinder to the connector labeled CHARGER on the side of the Pathfinder. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **PF-IBM cable**, not the Pathfinder's infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector on the top of the Pathfinder labeled with the picture of a **computer**.
3. Turn the computer On and insert the MTI CD disk into the appropriate drive. Use "Windows Explorer" to go to the drive that has the MTI CD disk in it or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD disk in it.
4. Double click on "MTI.exe". The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., Pathfinder).
 1. Go to the Pathfinder's Toolbox and open the TRANSFER MEMORY MENU.
 2. Activate the option you want to use (e.g., "Save Entire Device Contents" or "Save One User Area"). If everything is connected correctly, you will see the "Save to Memory File" on your computer screen.
 3. On your computer, go to the drive/open the folder where you want to save the memory.
 9. Type in a name for the new file, (e.g., PFMEM601). Click on "Save".
 10. The transfer could take awhile. When the transfer is finished, click, "OK".

To Restore a File:

1. Select "Load One User Area" or "Load Entire Device contents" from the TRANSFER MEMORY MENU.
2. On your computer, select the file you want to restore (e.g., PFMEM601).
3. Click on "Restore"

Put the MTI CD on your hard drive and create a desktop shortcut to it:

1. Using "Windows Explorer", go to the drive where you want to place the MTI CD files.
2. Create a new folder and label it "MTI".
3. Go to the drive that contains the MTI CD disk.
4. Right click on "MTI.exe".
5. Left click on "Copy".
6. Go to the drive where you created the new MTI folder. Right click on the folder.
7. Left click on "Paste".
8. Repeat these steps for the "MTI.hlp" file on the MTI CD disk.

➡ Instead of using "Copy" and "Paste", you can move the MTI CD files at once into the new folder if you want. Click on one of the files in the MTI CD disk, then hold down the SHIFT key and click the other MTI CD file. Release the SHIFT key and drag the highlighted files to your new MTI folder.

4. When the files have been moved, double click on the new MTI folder.
10. Right click on "MTI.exe".
11. Left click on "Create Shortcut".
12. The shortcut icon appears. Click on it and drag it to your desktop.
13. From now on you can open the MTI by double-clicking on this icon on your desktop.

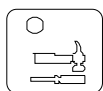
MTI for DOS

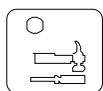
➡ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

➡ You can change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>

To Save Memory to a Computer Disk or Load it back into the Pathfinder:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your Pathfinder to the connector labeled CHARGER on the top of the Pathfinder. Plug the other end into a wall outlet.
 2. All memory transfers should be made with the **MTI-Serial-PF-to-IBM-PC cable**, not the device's infrared capability. Plug the one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled **SERIAL PORT** on the top of the Pathfinder.
 3. Turn the computer On and insert the MTI CD disk into the appropriate drive of your computer. Switch to the drive that has the disk in it if necessary.
- ➡ You may have to start your computer in DOS mode: start the computer, then go to the **Start** menu and click on **Shut Down**. Click on **Restart in DOS mode**. Restart the computer.
4. Type "DOSMTI.exe" and press <RETURN>.
 5. Type the letter of the device you are using: in this case type "P".



6. Press the  key on the left front of the Pathfinder case.
 7. Activate the TRANSFER MEMORY MENU in the Toolbox.
 8. Activate "Save Entire Device Contents" or "Save One User Area" to start the saving process.
- ➡ Use the "Load" options when you are loading vocabulary **into** your Pathfinder. Use "Merge" when you are adding new vocabulary to your current vocabulary..
9. On the computer, type in the name of the file you want to save the memory to (e.g., PFMEM1). Press F1 to complete the transfer.
- ➡ To **load** memory, select the file on the computer that you want to load the memory from.



Tips for a Successful MTI+ Memory Transfer (Macintosh)



Plug in the battery charger before you begin a memory transfer.



DO NOT attempt a memory transfer by using the infrared capability. Use the cable that came with your MTI+ disk and connect your Pathfinder and computer with the cable.

➤ The MTI+ does not work well on the following Macs running a system software version that is older than 7.5.3: Power Mac 6100, 7100 and 8100; Centris/Quadra 660av and 840av. You can update software from the following internet address: <http://asu.info.apple.com/>

➤ If you are using a Macintosh that only has a USB port, you will have to purchase a "USB-to-Serial" adapter. This will allow your MTI-MAC cable to work with the computer.

➤ If you are using an older Macintosh computer: turn the computer OFF. Make your connections. Hold the **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the Shift key.

➤ If you want to put the vocabulary on floppy disks, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➤ If you have **AppleTalk** turned On, you must turn it **Off** (inactive) before you can do a memory transfer. Make sure **SerialKeys** is **Off**.

➤ Once the transfer is in progress, do not plug anything into, or unplug anything from, the Pathfinder.

➤ When a transfer is in progress, periodically check the computer screen and Pathfinder display for any messages or problems.

➤ If you experience a problem, try changing the baud rate to 57600 in the MTI+ **Preferences** menu (inside the **Edit** menu).

➤ You may want to make a backup copy of each individual User Area as well as a backup copy of the entire device. This will prove helpful if you experience technical difficulties with your Pathfinder at some other time.

About Baud Rates

When the Pathfinder and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the Pathfinder must be the same.**

The default baud rate for the Pathfinderr is 57600. You can change the MTI baud rate to match the Pathfinder or you change the Pathfinder's baud rate to match the MTI.

To Change the MTI Baud Rate to Match the Pathfinder:

For the **MTI+**, click the **Edit** menu on the top of the computer display screen. Click **Preferences**. Click on the **Baud Rate** menu and select the rate you want.

To Change the Pathfinder's Baud Rate to Match the MTI:

Go to the Toolbox.

Activate the TRANSFER MEMORY MENU.

Select "More Items".

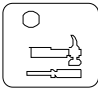
Activate the "MTI Baud Rate" box until you see the baud rate you want to use.

Activate OK.

Using the MTI+ for Macs

1. All memory transfers should be made with the **battery charger plugged in**. Plug the battery charger that came with your Pathfinder into the CHARGER connector on the side of the Pathfinder. Plug the other end into a wall outlet.
2. All memory transfers should be made using the **MTI+ cable** that came with your MTI+, not the Pathfinder's infrared capability. If you are using an **older Mac**, plug one end of the cable into either the connector on the back of the computer that has the picture of the telephone handset above it or the picture of a printer. Plug the other end into the connector that has the picture of a computer above it on the top of the Pathfinder. If you are using a **USB Mac**, plug the cable into the USB port on the computer. Plug the adapter and the other end of the cable into the connector that has the picture of a computer above it on the top of the Pathfinder.
3. Turn the computer On and insert the MTI CD. Double-click on the MTI+ icon to open the program.
4. **Copy the MTI+ to your Hard Drive for Better Performance:**
Click and hold on the folder labeled **MTI+**. Drag the folder to a location of your choice on your hard drive.
Close any open programs or applications. Then, double-click on the MTI+ folder on your hard drive to open the program.

Making a Transfer

5. Open the EDIT menu at the top of your computer screen and click on *Preferences*.
6. Make sure **Pathfinder** is selected in the **Device** menu.
7. Set the baud rate to the rate you want to use. The rate that is visible to begin with is the default baud rate for your selected device.
8. Select a computer **port** setting. If you are using an older Mac, select the port you connected your cable to (either Printer or Modem). If you are using a USB Mac, the instructions that came with your adapter should tell you which port to select. The Keyspan adapter requires you to use the **Printer** port.
9. Do not change any of the **Miscellaneous** settings.
10. Activate the  key on the left front of the Pathfinder case.
11. Activate TRANSFER MEMORY MENU.

12. Activate "Save Entire Device Contents" or "Save One User Area" to start the saving procedure.

➡ Use the "Load" option when you are loading vocabulary **into** your device. Use "Merge" if you are adding new vocabulary to your current vocabulary.

12. Type in the name of the file you want to use.

➡ To **load** memory, go to the correct drive (or disk) and double click on the file name you want. Click on "Continue".

➡ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ *Preferences* menu (inside the **Edit** menu). You may have to change the **port setting** on your computer. If you **are not** using a USB port, make sure the computer port you selected is the one your cable is plugged into, i.e., Modem or Printer. If you are using the **USB** port, the Keyspan adapter requires you to set the port to **Printer**. If you are using a different adapter, the information that came with the adapter should tell which computer port to set.

➡ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the Shift key.

Transfer Vocabulary from One Pathfinder to Another

When you plug the MTI-Serial-PF-to-IBM-PC cable into the SERIAL PORT connector on the top of each Pathfinder, you can transfer *vocabulary* from one Pathfinder to another. If you do not have the MTI-IBM cable, you can order it from PRC.

To Transfer Vocabulary from One Pathfinder to Another:

1. Plug the MTI-Serial-PF-to-IBM-PC cable into the SERIAL PORT connector on the back of each Pathfinder. (The ends of the cable are the same so it does not matter which end goes into which connector.)
2. Go to the Toolbox.
3. Activate the TRANSFER MEMORY on each Pathfinder. Make sure the baud rate is the same for both Pathfinders.
4. **On the Pathfinder that contains the vocabulary you want to *transfer*** , activate one of the "Save" options to begin the saving procedure.
5. "Looking for host computer...."
6. **On the Pathfinder that is going to receive the vocabulary**, activate one of the "Load" options to start the loading procedure.
5. The transfer takes place automatically.

Importing Graphics to the Pathfinder- Windows MTI Only

If you have the Windows MTI, you can import almost any graphic to use in the Pathfinder. You can import photographs that you have scanned, clip art from various web sites, graphics you have drawn using a graphics program, etc.

➡ If you do not have a graphics program on your computer, one is included with the MTI disk. This is a demonstration of Image Viewer/32™ and can only be used for a limited amount of time (30 days).

To import graphics, you need to remember two important rules:

- ➡ Graphics must be in the BMP format.
- ➡ Graphics must be in 256 colors.

If the MTI will not import a graphic, open a graphics program (either your own or the one included with the MTI), open the graphic you want to import and check the file format and number of colors for the graphic. If the graphic is not listed as a BMP file in 256 colors, save it using the BMP format and 256 colors.

An Example:

Let's say you have found a graphic on the web that you want to import to the Pathfinder.

1. Point to the graphic with your mouse and click on the **right** mouse button.
2. Click on "Save as" in the menu that pops up.
3. Select the drive and/or file where you want to save the graphic.
4. If the graphic is not in "BMP" format, click on "BMP" in the "Save as type" box. (If "BMP" is not an option in your menu save the graphic as it is.)

1. Change the title of the graphic if you want.

➡ Use up to 8 characters to name a graphic.

➡ Do not name your imported graphic with the same name as an original icon from the Pathfinder default icons. You will not be able to find your graphic once it is imported because the Pathfinder will only acknowledge the original icon.

6. Click on "Save".

You can do one of two things now:

➤ If you saved the graphic as a "BMP" file:

You can go ahead and open the MTI and try to import the graphic. See below for instructions. If the graphic was not in 256 colors, the MTI will not let you import it.

➤ If you could not save the graphic as a BMP file or if you have to check the colors:

1. Open your graphics program. Open the graphic that you just saved.
2. Open the menu that allows you to select color depth. In a graphics program like *PaintShop Pro*® this is the "Colors" menu. In other programs it might be under a different menu.
3. Set the color to 256. In *PaintShop Pro*® look at the "Increase Color Depth" and "Decrease Color Depth" options. If "256" is highlighted in either option, select it. If it isn't highlighted in either menu, the graphic was originally done in 256 colors and will transfer.
4. Go to the "File" menu and click on "Save as".
5. Select the drive and/or file where you want to save the graphic.
6. Select "BMP" from the "Save As Type" menu.
7. Change the title of the graphic if you want.
8. Click on "Save".

To Install and Use Image Viewer/32:

1. Put the **MTI** in the appropriate drive.
2. In the computer **Start** menu, select **Run**.
3. Type the **drive** you have the disk in (e.g., " a: ")
4. Type " **ivsetup** " (no spaces).
Example: a: ivsetup
5. See the installation screen on your computer.
6. Choose:
Next
Agree
Next
Next
Next
Next
Next
Finish
7. Activate **OK**. Activate **OK** again.

8. Go to the **Start** menu on your computer.
Select **Programs**.
Select **Arcata Pet Software**.
Select **IMGv3.2 Program File**
Select **OK**.
Open the program.
9. Find the photo or icon you want to use.
10. Select **Open**.
11. You should see your picture.
12. Open the **Image** menu in the menu bar.
13. Select **Convert To** and choose **256 colors**.
14. Open the **File** menu in the menu bar.
15. Select **Save As**.
16. Select **BMP**.
17. Select **Save**.

To Import the Graphic to the Pathfinder:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your Pathfinder to the connector labeled CHARGER on the top of the Pathfinder. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **PF-IBM cable**, not the Pathfinder's infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled **SERIAL PORT** on the top of the Pathfinder.
3. Turn the computer On and insert the MTI-IBM 95 disk into the appropriate drive. Use "Windows Explorer" to go to the drive that contains the MTI CD disk or right click on "My Computer" then click on "Explore" to go to the drive that has the disk in it.
4. Double click on the "MTI-EXE" icon. The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., Pathfinder).
6. Go to the Pathfinder's Toolbox and open the TRANSFER MEMORY MENU.
7. Activate "Import Icons". If everything is connected correctly, you will see the "Select one or more bitmap files to load" menu on your computer screen.
8. Go to the drive/open the folder where the graphic(s) you want to import is saved.
9. Click on the graphic you want to import.
10. Click on "Open".

11. If you want to change the name of the graphic, do it here. If you want the graphic to have different proportions, remove the "check" from the box.
12. Click "OK".
13. The transfer takes place. When it is finished, click on "OK".


To Check that the Graphic Transferred:

1. Go to the Pathfinder Toolbox.
2. Open the **ACTIVITY** menu.
3. Select **Create Activity**.
4. Type "**Test**" for a name.
5. Activate **More Items**.
6. You see a new key, "**IMPORTS**" in the **Icon Category** menu. Activate this key.
7. The graphic you imported should be visible and ready for use.
8. Activate **Cancel** to back your way out of the different **ACTIVITY** menu.

➡ Pathfinder places all imported graphics under the **IMPORTS** key in the **Icon Category** menu. Activate **Modify Item in Activity**, then activate **Change Icon** to get to the **IMPORTS** menu.

➡ **To delete a graphic from the Imports menu:** open the **MAINTENANCE** menu. Activate **Delete Imported Graphic**. Activate the graphic you want to delete. Answer **Yes**. Continue to delete graphics or **Exit** the menu.

Transferring Notebooks

 Make sure the Pathfinder battery charger is plugged in before transferring any Notebooks.

You can save your Notebooks on a disk or transfer text files from your computer to a Pathfinder Notebook using the MTI. You can open the Notebook on your computer and use a word processing program to add to/edit the text in your Notebook, if you wish. When you reload the Notebook from your computer into your Pathfinder, the Notebook will contain whatever changes you made to it.

To save a Notebook onto a Disk:

1. Connect the cable that came with your MTI to your Computer and your Pathfinder. Turn your computer On and insert the MTI CD. Open the MTI program.
2. Select Pathfinder from the **Options** menu.
3. Select the **Save to Computer** icon on the menu.
4. Activate NOTEBOOK MENU on the Pathfinder.
5. Go to Screen 2 and activate option 6 (*Save Notebook to Disk*).
6. Open the Notebook you want to save by choosing it from the menu.
7. The Pathfinder will search for the host computer. If the connections between the Pathfinder and your computer are correct, the computer will ask you to type in the name of the file (Notebook) you want to save. Do so, and then continue to follow the instructions on your computer screen.

Follow the same format for loading a Notebook (or computer file) back into your Pathfinder, only this time choose option 5 from the NOTEBOOK MENU. Follow the prompts on the Pathfinder and the computer display screens.

Note:

Notebooks or files in your computer which you want to move to the Pathfinder **must** be saved on the computer in **TEXT ONLY** format

Importing WAV (Sound) Files to the Pathfinder: Windows MTI Only

In order to transfer wav files to your device, you must first have them stored on your computer. An easy way to find wav files is to type "wav files" into a search engine on your internet browser, then save a few files to a new file on your computer. ⤷ **The files you save must be in PCM format.** To check this: right click on the wav file you want, click on "Properties", then "Summary" to see the format.

To Transfer a Wav File:

1. All memory transfers should be made with the **battery charger plugged in.** Connect one end of the battery charger that came with your Pathfinder to the connector labeled CHARGER on the side of the Pathfinder. Plug the other end into a wall outlet.
 2. All memory transfers should be made with the **PF-IBM cable**, not the Pathfinder's infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the computer serial port connector on the top of the Pathfinder that is next to the Speaker and Earphone connectors.
 3. Turn the computer On and insert the MTI CD disk into the appropriate drive. Use "Windows Explorer" to go to the drive that has the MTI CD disk in it or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD disk in it.
 4. Double click on "MTI.exe". The MTI screen is visible on your computer monitor.
 5. In the MTI "Options" menu and click on the name of the device you are using.
 6. Go to the Pathfinder's Toolbox and select the **STORE CORE** key (you can save wav files using any key or menu which allows you to store vocabulary). .
 7. Activate the icon or icons you want to store the wav file under, then activate **OK**.
 8. Select the **Recorded** option.
 9. Select the **Import Sound File** option.
 10. On your computer, go to the drive/open the folder where you have your wav files saved.
 11. Select the file you want to import and click on "Open".
 12. You will see a **Memory Transfer Complete** message box when the file has been transferred. Click on **OK** in the message box. You can select **Listen to Recording** to check that you got the correct file. The wav file is now the recorded message for that icon or icon sequence.
 13. Now you can change the icon, the label, the key color, etc., if you wish to.
- ⤷ If you have a previously recorded message assigned to a key or sequence and you import a wav file, the wav file will **replace** the recorded message. Any macros assigned to the key or sequence will remain in place.

Memory Transfers Using a Secure Digital Card Or USB Memory Device—Pathfinder Plus Only

The Pathfinder with Plus features allows you to transfer files from the Pathfinder to a computer or computer to Pathfinder by using a Secure Digital card or a USB memory device (sometimes called a "Keychain" or "Keyfob" device).

Secure Digital Card Transfer

1. Insert the Secure Digital card in the small slot above the Mouse Serial Port.. Insert the card face-up (the side with the writing on it should be face up). Push it in until it stays securely. (Push on the end of the card again to remove the card.)
2. Go to the **TOOLBOX** and open the **TRANSFER MEMORY MENU**.
3. Change the *MTI Method* option to **SD Card**. If the card is securely in place, you will see a note telling you *Disk Status: READY* and the amount of space available on the card.

➡ **It may take a few seconds** for the Option name to change in the box after you select an up or down arrow.

4. Now select the memory transfer option you want to use and follow the directions you see on the display screen.
5. You will see a message box telling you when the transfer is completed.

USB Transfer

1. Connect the USB memory device to the **rectangular** USB port on the left side of the Pathfinder case.
2. Go to the **TOOLBOX** and open the **TRANSFER MEMORY MENU**.
3. Change the *MTI Method* option to **USB Disk**. If the USB device is securely in place, you will see a note telling you *Disk Status: READY* and the amount of space available on the card.

➡ **It may take a few seconds** for the Option name to change in the box after you select an up or down arrow.

4. Now select the memory transfer option you want to use and follow the directions you see on the display screen.
5. You will see a message box telling you when the transfer is completed.

➡ IMPORTANT NOTE

Importing Sound (wav) Files and Saving/Loading Notebooks

You import sound files while you are in storing menus and you save or load Notebooks while you are in the NOTEBOOK MENU. After connecting your memory device, **it is a good idea to go first** to the MEMORY TRANSFER MENU and select either **SD Card** or **USB Disk**. Then begin the storing process for importing sound files or go to the NOTEBOOK MENU to load or save Notebooks.

**MTI for the
Vantage and Vanguard II
Including Plus Products**



Tips for a Successful Windows or DOS Memory Transfer



Plug in the battery charger before you begin a memory transfer.



DO NOT attempt a memory transfer by using the infrared capability. Use the VAN/PF cable that came with your MTI disks and connect your device and computer with this cable.

➡ If you are going to save vocabulary to an IBM disk, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➡ Make sure **SerialKeys** is turned Off.

➡ Saving **One User Area** will **not** save any changes you may have made to the Word Prediction Dictionary and it will **not** save any IR commands the Vantage or Vanguard II may have learned. You must use **Save Entire Device** to save IR commands and Word Prediction Dictionary changes.

➡ If you are going to **download** vocabulary that was programmed for a 15, 8 or 4 location overlay, make sure you are currently using the 15, 8 or 4 overlay on your device. If you aren't, you must open the CHANGE OVERLAY MENU and switch to the 15, 8 or 4 keyboard and overlay **before** you download the vocabulary.

➡ Before you begin a transfer with this MTI, remove any older versions of the MTI that you may have on your computer

➡ Once the transfer is in progress, do not plug anything into, or unplug anything from, the Vantage or Vanguard II.

➡ When a transfer is in progress, periodically check the computer screen and device display for any messages or problems.

➡ If you experience a problem, try changing the baud rate to 57600 in the MTI menu bar.

About Baud Rates

When the Vantage or Vanguard II and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the Vantage must be the same.**

The default baud rate for the Vantage or Vanguard II is 57600. You can change the MTI baud rate to match your device or you change the device's baud rate to match the MTI program.

To Change the MTI Baud Rate to Match the Vantage or Vanguard II:

If you are using DOS, change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>.

With Windows, change the baud rate in the MTI "Options" menu.

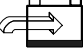

To Change the Vantage or Vanguard II Baud Rate to Match the MTI:

1. Go to the Toolbox.
2. Activate the TRANSFER MEMORY MENU.
3. Activate the arrows in the "MTI Baud Rate" boxes until you see the baud rate you want to use.
4. Activate OK.

Save Vocabulary on an IBM Computer Using the MTI for Windows[®]

➡ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you begin.

To save memory to an IBM computer running Windows or to load memory back into the Vantage or Vanguard II:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your device to the connector labeled  on the communication device. Plug the other end into a wall outlet.
2. All memory transfers should be made with the VG-IBM cable, not the infrared capability. Plug one end of this cable into Serial Port 1 on your computer. **On the Vantage:** plug the other end into the connector labeled  on the side of the case. **On the Vanguard II:** use the connector with the computer icon and the letter **A** next to it.
3. Turn the computer On and insert the MTI CD into the appropriate drive. Use "Windows Explorer" to go to the drive that has the MTI CD in it or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD in it.
4. Double click on "MTI.exe". The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., Vantage or Vanguard II).
6. Go to the Toolbox and open the TRANSFER MEMORY MENU.
7. Activate the option you want to use (e.g., "Save Entire Device Contents" or "Save One User Area"). If everything is connected correctly, you will see the "Save to Memory File" on your computer screen.
8. On your computer, go to the drive/open the folder where you want to save the memory.
9. Type in a name for the new file, (e.g., AMYMEM4-03). Click on "Save".
10. The transfer could take awhile. When the transfer is finished, click, "OK".

To Restore a File:

1. Select "Load Entire Device" or "Load One User Area" from the TRANSFER MEMORY MENU.
2. On your computer, select the file you want to restore (e.g., VTMEM601).
3. Click on "Restore"

Put the MTI on your hard drive and create a desktop shortcut to it:

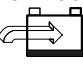

1. Using "Windows Explorer", go to the drive where you want to place the MTI files.
 2. Create a new folder and label it "MTI".
 3. Go to the drive that contains the MTI CD.
 4. Right click on "MTI.exe".
 5. Left click on "Copy".
 6. Go to the drive where you created the MTI folder. Right click on the folder.
 7. Left click on "Paste".
 8. Repeat these steps for the "MTI.hlp" file on the MTI CD disk.
- ➡ Instead of using "Copy" and "Paste", you can move the MTI files at once into the new folder if you want. Click on one of the files in the MTI CD disk, then hold down the SHIFT key and click the other MTI CD file. Release the SHIFT key and drag the highlighted files to your new MTI folder.
9. When the files have been moved, double click on the new MTI folder.
 10. Right click on "MTI.exe".
 11. Left click on "Create Shortcut".
 12. The shortcut icon appears. Click on it and drag it to your desktop.
 13. From now on you can open the MTI by double-clicking on this icon on your desktop.

MTI for DOS

➤ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

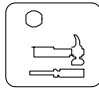
➤ You can change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>

To Save Memory to a Computer Disk or Load it back into the Vantage or Vanguard II:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your device to the connector labeled . Plug the other end into a wall outlet.
2. All memory transfers should be made with the VG-IBM cable, not the infrared capability. Plug one end of this cable into Serial Port 1 on your computer. **On the Vantage:** plug the other end into the connector labeled  on the side of the case. **On the Vanguard II:** use the connector with the computer icon and the letter **A** next to it.
3. Turn the computer On and insert the MTI CD into the appropriate drive. Switch to the drive that has the MTI CD in it if necessary.

➤ You may have to start your computer in DOS mode: start the computer, then go to the **Start** menu and click on **Shut Down**. Click on **Restart in DOS mode**. Restart the computer.

4. Type "DOSMTI.exe" and press <RETURN>.
5. Type the letter of the device you are using: in this case the "V" will work with Vantage, Vanguard and Vanguard 11.

6. Press the  key on the front of the case.
You see the Toolbox screen.

7. Activate the TRANSFER MEMORY MENU in the Toolbox.
8. Activate "Save Entire Device Contents" or "Save One User Area" to start the saving process.

➤ Use "Load Entire Device" or "Load One User Area" when you are loading vocabulary **into** your device. Use "Merge" when you are downloading a MAPMate.

9. On the computer, type in the name of the file you want to save the memory to (e.g., VGMEM1). Press F1 to complete the transfer.

➤ To **load** memory, select the file on the computer that you want to load the memory from.



Tips for a Successful MTI+ Memory Transfer (Macintosh)



Plug in the battery charger before you begin a memory transfer.



DO NOT attempt a memory transfer by using the infrared capability. Use the cable that came with your MTI+ and connect your Vantage or Vanguard II and computer with this cable.

➤ The MTI+ does not work well on the following Macs running a system software version that is older than 7.5.3: Power Mac 6100, 7100 and 8100; Centris/Quadra 660av and 840av. You can update software from the following internet address: <http://asu.info.apple.com/>

➤ If you are using a Macintosh that only has a USB port, you will have to purchase a "USB-to-Serial" adapter. PRC has tested the Vantage and Vanguard II using an adapter made by Keyspan. Check the Keyspan web page or do a search for "USB to Serial Adapters".

➤ If you want to put the vocabulary on floppy disks, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➤ If you have **AppleTalk** turned On, you must turn it **Off** (inactive) before you can do a memory transfer. Make sure **SerialKeys** is **Off**.

➤ Once the transfer is in progress, do not plug anything into, or unplug anything from, the Vantage.

➤ When a transfer is in progress, periodically check the computer screen and device display for any messages or problems.

➤ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ *Preferences* menu (inside the **Edit** menu). You may have to change the **port setting**. If you are **not** using a USB port, set the port setting to match the port your cable is in (Modem or Printer). The **Keyspan adapter** requires the port to be set to **Printer** to work.

About Baud Rates

When the Vantage or Vanguard II and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the Vantage or Vanguard II must be the same.**

The default baud rate for your communication device is 57600. You can change the MTI baud rate to match the device or you change the device's baud rate to match the computer's.

To Change the MTI Baud Rate to Match the Vantage or Vanguard II:

For the MTI+, click the **Edit** menu on the top of the computer display screen. Click **Preferences**. Click on the **Baud Rate** menu and select the rate you want.

To Change the Vantage or Vanguard II Baud Rate to Match the MTI:


Go to the Toolbox.

Activate the TRANSFER MEMORY MENU.

Activate the arrows in the "MTI Baud Rate" boxes until you see the baud rate you want to use.

Activate OK.

Using the MTI+ for Macs

1. All memory transfers should be made with the **battery charger plugged in**. Plug the battery charger that came with your device into the CHARGER connector on the device. Plug the other end into a wall outlet.
2. **All memory transfers should be made using the MTI+ cable** that came with your MTI+. If you are using an older Mac, plug one end of the cable into the connector on the back of the computer that has the picture of the telephone handset above it or the picture of a printer. **On the Vantage:** plug the other end into connector on the side of the case that has the picture of a computer by it. **On the Vanguard II:** plug the other end of the cable into the connector that has a computer icon and the letter **A** next to it. If you are using a USB adapter, plug the adapter with its cable into the USB port on your computer. Plug the MTI cable into the other end of the adapter and then plug it into the connector that has the picture of a computer by it on the Vantage or the computer icon with the letter **A** on the Vanguard II.
3. Turn the computer On and insert the MTI CD. Double-click on the icon to open the program.
4. **Copy the MTI+ to your Hard Drive for Better Performance:**
Click and hold on the folder labeled **MTI+**. Drag the folder to a location of your choice on your hard drive.
Close any open programs or applications. Then, double-click on the MTI+ folder on your hard drive to open the program.
5. Open the EDIT menu at the top of your computer screen and click on *Preferences*.
6. Make sure **Vantage or Vanguard II** is selected in the **Device** menu.
7. Set the **baud rate** to the rate you want to use. The rate that is visible to begin with is the default baud rate for your selected device.
8. Select a **Computer Port**: If you are using an older Mac: select the port you plugged your cable into, either Printer or Modem. If you are using a USB adapter, the adapter must be connected to the serial port on the on your device. **On the Vantage** this is the connector with the picture of the computer by it. **On the Vanguard II** this is the connector with the computer icon and the letter **A** beside it. Select the computer port that your adapter instructions tell you to. If you are using a **Keyspan** adapter, use the **printer port**.
9. Do not change any of the **miscellaneous** settings.
10. Activate the  key on the front of the case.

11. Activate TRANSFER MEMORY MENU.

12. Activate one of the **Save** options to start the saving procedure.

➡ Use the "Load" option when you are loading vocabulary **into** your device. You will be asked if you want to merge the vocabulary you are loading with your current vocabulary or if you want to delete the current vocabulary. If you select **Merge**, the device will give precedence to the vocabulary you are loading. If you choose **Delete**, the vocabulary already in the Area will be deleted first, then the new vocabulary loaded.

13. Type in the name of the file you want to use.

➡ To **load** memory, go to the correct drive (or disk) and double click on the file name you want. Click on "Continue".

➡ **If you experience a problem**, try changing the baud rate for the computer to 57600 in the MTI+ *Baud Rate* menu under *Preferences* in the **Edit** menu on the menu bar. Remember, the baud rate for your device and the computer must be the same. Also, make sure the correct computer port has been used and selected.

➡ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the Shift key.

Transfer Vocabulary from One Device to Another

When you plug the **MTI-Serial-VG-to-IBM-PC** cable into the SERIAL PORT connector on the each Vantage or Vanguard II, you can transfer *vocabulary* from one device to another. If you do not have the MTI-IBM cable, you can order it from PRC.

To Transfer Vocabulary from One Vantage to Another Vantage or from one Vanguard II to another Vanguard II:


1. Plug the MTI-Serial-VG-to-IBM-PC cable into the connector with the picture of a computer by it on the side of each **Vantage**. Use the port with the computer icon and the letter **A** beside it on the **Vanguard II**. (The ends of the cable are the same so it does not matter which end goes into which connector.)
2. Go to the Toolbox.
3. Activate the TRANSFER MEMORY Menu on each device. (Make sure the baud rate is the same for both each device in the TRANSFER MEMORY MENUS).
4. **On the device that contains the vocabulary you want to transfer**, activate one of the **Save** options to begin the saving procedure.
5. **On the device that is going to receive the vocabulary**, activate one of the **Load** options to start the loading procedure.
6. The transfer takes place automatically.

➡ If you want to transfer specific Activities from one device to another, **first** load the Activities into a blank User Area in the device. Then save and load that Area.

➡ **REMEMBER:** If you save a single User Area, the Vantage or Vanguard II will not save any changes you may have made in the Word Prediction Menu, and it will not save any IR commands it may have learned. Use **Save Entire Device** to save IR commands and Dictionary changes.

➡ **You can also transfer an Area or the Entire Device Contents from a Vantage to the Vanguard II or vice versa.** Activate the *Save as Vanguard (or Vantage)* option in the TRANSFER MEMORY MENU until **On** is visible in the box. Perform a regular MTI to a computer disk. Then use the MTI to load the file from the computer into the Vanguard or Vantage.

Transferring Notebooks

 Make sure the battery charger is plugged in before transferring any Notebooks.

You can save your Notebooks on a disk or transfer text files from your computer to your device Notebook using the MTI. You can open the Notebook on your computer and use a word processing program to add to/edit the text in your Notebook, if you wish. When you reload the Notebook from your computer into your device, the Notebook will contain whatever changes you made to it.

To save a Notebook onto a Disk:

1. Connect the cable that came with your MTI to your Computer and your device. Turn your computer On and insert the MTI CD. Open the MTI program.
4. Select your device name from the **Options** menu.
5. Select the **Save to Computer** icon on the menu.
4. Activate **NOTEBOOK MENU** in the **TOOLBOX**.
5. Activate *Save Notebook to Disk*.
6. Open the Notebook you want to save by choosing it from the menu.
7. If the connections between your device and your computer are correct, the computer will ask you to type in the name of the file (Notebook) you want to save. Do so, and then continue to follow the instructions on your computer screen.

Follow the same format for loading a Notebook (or computer file) back into your device, only this time choose option 5 from the NOTEBOOK MENU. Follow the prompts on the device display and the computer display screens.

Note:

Notebooks or files in your computer which you want to transfer to your AAC device **must** be saved on the computer in **TEXT ONLY format**

Importing Graphics – Windows® MTI Only

If you have the Windows MTI, you can import almost any graphic to use in the Vantage or Vanguard II. You can import photographs that you have scanned, clip art from various web sites, graphics you have drawn using a graphics program, etc.

To import graphics, you need to remember two important rules:

- ➡ Graphics must be in the BMP format.
- ➡ Graphics must be in 256 colors.

If the MTI will not import a graphic, open your graphics program, open the graphic you want to import and check the file format and number of colors for the graphic. If the graphic is not listed as a BMP file in 256 colors, save it using the BMP format and 256 colors.

➡ If you do not have a graphics program on your computer, one is included on the MTI disk. This is a demonstration of Image Viewer/32™ and can only be used for a limited amount of time (30 days).

An Example:

Let's say you have found a graphic on the web that you want to import to your device.

1. Point to the graphic with your mouse and click on the **right** mouse button.
2. Click on "Save as" in the menu that pops up.
3. Select the drive and/or file where you want to save the graphic.
4. If the graphic is not in "BMP" form, click on "BMP" in the "Save as type" box. (If "BMP" is not an option in your menu save the graphic as it is.)

1. Change the title of the graphic if you want.

➡ Use up to 8 characters to name a graphic.

➡ Do not name your imported graphic with the same name as an original icon from the Vantage or Vanguard II default icons. You will not be able to find your graphic once it is imported because your device will only acknowledge the original icon.

6. Click on "Save".

You can do one of two things now:

➡ **If you saved the graphic as a "BMP" file:**

You can go ahead and open the MTI and try to import the graphic. See below for instructions. If the graphic was not in 256 colors, the MTI will not let you import it.

➡ **If you could not save the graphic as a BMP file or if you have to check the colors:**


1. Open your graphics program. Open the graphic that you just saved.
2. Open the menu that allows you to select color depth. In a graphics program like *PaintShop Pro*® this is the "Colors" menu. In other programs it might be under a different menu.
3. Set the color to 256. In *PaintShop Pro*®, look at the "Increase Color Depth" and "Decrease Color Depth" options. If "256" is highlighted in either option, select it. If it isn't highlighted in either menu, the graphic was originally done in 256 colors and will transfer.
4. Go to the "File" menu and click on "Save as".
5. Select the drive and/or file where you want to save the graphic.
6. Select "BMP" from the "Save As Type" menu.
7. Change the title of the graphic if you want.
8. Click on "Save".

To Install and Use Image Viewer/32:

1. Put the **MTI CD** in the appropriate drive.
2. In the computer **Start** menu, select **Run**.
3. Type the **drive** you have the disk in (e.g., " a: ")
4. Type "**ivsetup** " (no spaces).
Example: a: ivsetup
5. See the installation screen on your computer.
6. Choose:
Next
Agree
Next
Next
Next
Next
Next
Finish
7. Activate **OK**. Activate **OK** again.
8. Go to the **Start** menu on your computer.
Select **Programs**.
Select **Arcata Pet Software**.
Select **IMGv3.2 Program File**
Select **OK**.
Open the program.

9. Find the photo or icon you want to use.
10. Select **Open**.
11. You should see your picture.
12. Open the **Image** menu in the menu bar.
13. Select **Convert To** and choose **256 colors**.
14. Open the **File** menu in the menu bar.
15. Select **Save As**.
16. Select **BMP** and name the graphic.
17. Select **Save**.

To Import the Graphic to the Vantage or Vanguard II:


1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your device to the connector labeled CHARGER on your device. Plug the other end into a wall outlet.
2. All memory transfers should be made with the VG-IBM cable, not the infrared capability. Plug one end of this cable into Serial Port 1 on your computer. **On the Vantage:** plug the other end into the connector labeled  on the side of the case. **On the Vanguard II:** use the connector with the computer icon and the letter **A** next to it.
3. Turn the computer On and insert the MTI CD disk into the appropriate drive of your computer. Use "Windows Explorer" to go to the drive that contains the MTI CD or right click on "My Computer" then click on "Explore" to go to the drive that has the disk in it.
4. Double click on the "MTI-EXE" icon. The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., Vantage or Vanguard II).
6. Go to the Toolbox and open the TRANSFER MEMORY MENU.
7. Activate "Import Icons". If everything is connected correctly, you will see the "Select one or more bitmap files to load" menu on your computer screen.
8. Go to the drive/open the folder where the graphic(s) you want to import is saved.
9. Click on the graphic you want to import. (If you want to import more than one graphic, hold down the CONTROL key and click on each graphic that you want to import.)
10. Click on "Open".
11. If you want to change the name of the graphic, do it here. If you want the graphic to have different proportions, remove the "check" from the box.

12. Click "OK".

13. The transfer takes place. When it is finished, click on "OK".

To Check that the Graphic Transferred:

1. Go to the Toolbox.
2. Open the MODIFY PAGE MENU.
3. Select any page, e.g., "Jokes".
4. Activate an unused key.
5. Activate "Change Icon".
6. You see a new key, "IMPORTS" in the Icon Category menu. Activate this key.
7. The graphic you imported should be visible and ready for use.
8. Activate "Cancel" to back your way out of the different MODIFY PAGE menus.
9. If you have graphics that you no longer want in this menu, activate "Delete Icon" and select the graphic to be deleted.

 All imported graphics are placed under the **IMPORTS** key in the *Icon Category* menu. Activate MODIFY ACTIVITY or MODIFY PAGE to access this menu.

Importing Wav (Sound) Files: Windows MTI Only

In order to transfer wav files to your device, you must first have them stored on your computer. An easy way to find wav files is to type "wav files" into a search engine on your internet browser, then save a few files to a new folder on your computer. ➡ **The files you save must be in PCM format.** To check this: right click on the wav file you want, click on "Properties", then "Summary" to see the format.

To Transfer a Wav File:

1. All memory transfers should be made with the **battery charger plugged in.** Connect one end of the battery charger that came with your Pathfinder to the connector labeled CHARGER on the side of the Pathfinder. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **VG-IBM cable**, not the infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the computer port (Vantage) or computer port A (Vanguard II).
3. Turn the computer On and insert the MTI CD disk into the appropriate drive. Use "Windows Explorer" to go to the drive that has the MTI CD disk in it or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD disk in it.
4. Double click on "MTI.exe." The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the name of the device you are using.
4. Go to your device's Toolbox and select the **STORE CORE** key (you can save wav files using any key or menu which allows you to store vocabulary). .
5. Activate the icon or icons you want to store the wav file under, then activate **OK**.
6. Select the **Recorded** option.
7. Select the **Import Sound File** option.
8. On your computer, go to the drive/open the folder where you have your wav files saved.
11. Select the file you want to import and click on "Open".
12. You will see a **Memory Transfer Complete** message box when the file has been transferred. Click on **OK** in the message box. You can select **Listen to Recording** to check that you got the correct file. The wav file is now the recorded message for that icon or icon sequence.
13. Now you can change the icon, the label, the key color, etc., if you wish to.

➡ If you have a previously recorded message assigned to a key or sequence and you import a wav file, the wav file will **replace** the recorded message. Any macros assigned to the key or sequence will remain in place.

Memory Transfers Using a Secure Digital Card Or USB Memory Device—Vantage Plus Only

The Vantage with Plus features allows you to transfer files from the Vantage to a computer or computer to Vantage by using a Secure Digital card or by using a USB memory device (sometimes called a "Keychain" or "Keyfob" device).

Secure Digital Card Transfer

6. Open the Memory door on the left side of the Vantage and insert the Secure Digital card in the small slot above the system card. Insert the card face-up (the side with the writing on it should be face up). Push it in until it stays securely. (Push on the end of the card again to remove the card.)
7. Go to the **TOOLBOX** on the Vantage and open the **TRANSFER MEMORY MENU**.
8. Change the *MTI Method* option to *SD Card*. If the card is securely in place, you will see a note telling you *Disk Status: READY* and the amount of space available on the card.

➡ **It may take a few seconds** for the Option name to change in the box after you select an up or down arrow.

9. Now select the memory transfer option you want to use and follow the directions you see on the display screen.
10. You will see a message box telling you when the transfer is completed.

USB Transfer

1. Connect the USB memory device to the **rectangular** USB port on the right side of the Vantage case.
2. Go to the **TOOLBOX** and open the **TRANSFER MEMORY MENU**.
3. Change the *MTI Method* option to *USB Disk*. If the USB device is securely in place, you will see a note telling you *Disk Status: READY* and the amount of space available on the card.

➡ **It may take a few seconds** for the Option name to change in the box after you select an up or down arrow.

4. Now select the memory transfer option you want to use and follow the directions you see on the display screen.
5. You will see a message box telling you when the transfer is completed.

⚠ IMPORTANT NOTE

Importing Sound (wav) Files and Saving/Loading Notebooks

You import sound files while you are in storing menus and you save or load Notebooks while you are in the **NOTEBOOK MENU**. After connecting your memory device, **it is a good idea to go first** to the **MEMORY TRANSFER MENU** and select either *SD Card* or *USB Disk*. Then begin the storing process for importing sound file or go to the **NOTEBOOK MENU** to save or load Notebooks.

MTI for the SpringBoard



Tips for a Successful Windows or DOS Memory Transfer



Plug in the battery charger before you begin a memory transfer.



Use the MTI cable that came with your MTI and connect your SpringBoard and computer with the cable.

➤ If you are going to save vocabulary to an IBM disk, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➤ If you are going to **download** vocabulary that was programmed for a 15, 8 or 4 location overlay, make sure you are currently using the 15, 8 or 4 overlay in the SpringBoard. If you aren't, you must open the CHANGE OVERLAY MENU and switch to the 15, 8 or 4 keyboard and overlay **before** you download the vocabulary.

➤ Before you begin a transfer with this MTI, remove any older versions of the MTI that you may have on your computer

➤ Once the transfer is in progress, do not plug anything into, or unplug anything from, the SpringBoard.

➤ When a transfer is in progress, periodically check the computer screen and SpringBoard display for any messages or problems.

➤ If you experience a problem, try changing the baud rate to 57600 in the MTI menu bar.

About Baud Rates

When the SpringBoard and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the SpringBoard must be the same.**

The default baud rate for the SpringBoard is 57600. You can change the MTI baud rate to match the SpringBoard or you change the SpringBoard's baud rate to match the MTI program.

To Change the MTI Baud Rate to Match the SpringBoard:

If you are using DOS, change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 57600 <ENTER>.

With Windows, change the baud rate in the MTI "Options" menu.

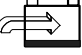

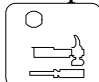
To Change the SpringBoard's Baud Rate to Match the MTI:

1. Go to the Toolbox.
2. Activate the TRANSFER MEMORY MENU.
3. Activate the arrows in the "MTI Baud Rate" boxes until you see the baud rate you want to use.
4. Activate OK.

Save Vocabulary on an IBM Computer Using the MTI for Windows[®]

➡ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you begin.

To save memory to an IBM computer running Windows or to load memory back into the SpringBoard:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your SpringBoard to the connector labeled  on the right side of the SpringBoard. Plug the other end into a wall outlet.
2. All memory transfers should be made with the MTI Serial cable that came with your MTI. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled  on the left side of the SpringBoard.
3. Turn the computer On and insert the MTI CD into the appropriate drive. Use "Windows Explorer" to go to the drive that has the MTI CD in it or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD in it.
4. Double click on "MTI.exe". The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., SpringBoard).
6. Press the  key on the front of the SpringBoard case. You see the Toolbox screen.
7. Activate the option you want to use (e.g., "Save Entire Device Contents" or "Save One User Area"). If everything is connected correctly, you will see the "Save to Memory File" on your computer screen.
8. On your computer, go to the drive/open the folder where you want to save the memory.
9. Type in a name for the new file, (e.g., SBMEM1). Click on "Save".
10. The transfer could take awhile. When the transfer is finished, click, "OK".

To Restore a File:

1. Select "Load Entire Device" or "Load One User Area" from the TRANSFER MEMORY MENU.
2. On your computer, select the file you want to restore (e.g., VTMEM601).
3. Click on "Restore"

Put the MTI on your hard drive and create a desktop shortcut to it:



2. Using "Windows Explorer", go to the drive where you want to place the MTI files.
2. Create a new folder and label it "MTI".
3. Go to the drive that contains the MTI CD.
4. Right click on "MTI.exe".
5. Left click on "Copy".
6. Go to the drive where you created the MTI folder. Right click on the folder.
7. Left click on "Paste".
8. Repeat these steps for the "MTI.hlp" file on the MTI CD disk.
- ➡ Instead of using "Copy" and "Paste", you can move the MTI files at once into the new folder if you want. Click on one of the files in the MTI CD disk, then hold down the SHIFT key and click the other MTI CD file. Release the SHIFT key and drag the highlighted files to your new MTI folder.
9. When the files have been moved, double click on the new MTI folder.
10. Right click on "MTI.exe".
11. Left click on "Create Shortcut".
12. The shortcut icon appears. Click on it and drag it to your desktop.
13. From now on you can open the MTI by double-clicking on this icon on your desktop.

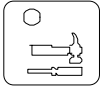
MTI for DOS

➤ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

➤ You can change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>

To Save Memory to a Computer Disk or Load it back into the SpringBoard:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your SpringBoard to the connector labeled  on the right side of the SpringBoard. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **MTI cable** that came with your MTI. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled  on the left side of the SpringBoard.
3. Turn the computer On and insert the MTI CD into the appropriate drive. Switch to the drive that has the MTI CD in it if necessary.
4. Type "DOSMTI.exe" and press <RETURN>.
5. Type the letter of the device you are using: in this case type "V".

6. Press the  key on the front of the SpringBoard case. You see the Toolbox screen.

7. Activate the TRANSFER MEMORY MENU in the Toolbox.
8. Activate "Save Entire Device Contents" or "Save One User Area" to start the saving process.

➤ Use "Load Entire Device" or "Load One User Area" when you are loading vocabulary **into** your SpringBoard. Use "Merge" when you are downloading a MAPMate.

9. On the computer, type in the name of the file you want to save the memory to (e.g., SBMEM1). Press F1 to complete the transfer.

➤ To **load** memory, select the file on the computer that you want to load the memory from.



Tips for a Successful MTI+ Memory Transfer (Macintosh)



Plug in the battery charger before you begin a memory transfer.



DO NOT attempt a memory transfer by using the infrared capability. Use the cable that came with your MTI+ and connect your SpringBoard and computer with the cable.

➤ The MTI+ does not work well on the following Macs running a system software version that is older than 7.5.3: Power Mac 6100, 7100 and 8100; Centris/Quadra 660av and 840av. You can update software from the following internet address: <http://asu.info.apple.com/>

➤ If you are using a Macintosh that only has a USB port, you will have to purchase a "USB-to-Serial" adapter. PRC has tested the SpringBoard using an adapter made by Keyspan. Check the Keyspan web page or do a search for "USB to Serial Adapters".

➤ If you want to put the vocabulary on floppy disks, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➤ If you have **AppleTalk** turned On, you must turn it **Off** (inactive) before you can do a memory transfer. Make sure **SerialKeys** is turned **Off**.

➤ Once the transfer is in progress, do not plug anything into, or unplug anything from, the SpringBoard.

➤ When a transfer is in progress, periodically check the computer screen and SpringBoard display for any messages or problems.

➤ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ **Preferences** menu (inside the **Edit** menu). You may have to change the **port setting**. If you are **not** using a USB port, set the port setting to match the port your cable is in (Modem or Printer). The **Keyspan adapter** requires the port to be set to **Printer** to work.

➤ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the Shift key.

About Baud Rates

When the SpringBoard and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the SpringBoard must be the same.**

The default baud rate for the SpringBoard is 57600. You can change the MTI baud rate to match the SpringBoard or you change the SpringBoard's baud rate to match the computer's.

To Change the MTI Baud Rate to Match the SpringBoard:

For the **MTI+**, click the **Edit** menu on the top of the computer display screen. Click **Preferences**. Click on the **Baud Rate** menu and select the rate you want.

To Change the SpringBoard's Baud Rate to Match the MTI:

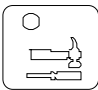
Go to the Toolbox.

Activate the TRANSFER MEMORY MENU.

Activate the arrows in the "MTI Baud Rate" boxes until you see the baud rate you want to use.

Activate OK.

Using the MTI+ for Macs

1. All memory transfers should be made with the **battery charger plugged in**. Plug the battery charger that came with your SpringBoard into the CHARGER connector on the SpringBoard. Plug the other end into a wall outlet.
2. **All memory transfers should be made using the MTI+ cable** that came with your MTI+. If you are using an older Mac, plug one end of the cable into the connector on the back of the computer that has the picture of the telephone handset above it or the picture of a printer. If you are using a USB adapter, plug the adapter cable into the USB port. Plug the MTI cable into the adapter and the other end of the cable into the connector that has the picture of a computer by it on the SpringBoard.
3. Turn the computer On and insert the MTI CD. Double-click on the icon to open the program.
4. **Copy the MTI+ to your Hard Drive for Better Performance:**
Click and hold on the folder labeled **MTI+**. Drag the folder to a location of your choice on your hard drive.
Close any open programs or applications. Then, double-click on the MTI+ folder on your hard drive to open the program.
5. Open the EDIT menu at the top of your computer screen and click on *Preferences*.
6. Make sure **SpringBoard** is selected in the **Device** menu.
7. Set the **baud rate** to the rate you want to use. The rate that is visible to begin with is the default baud rate for your selected device.
8. Select a **Computer Port**: If you are using an older Mac: select the port you plugged your cable into, either Printer or Modem. If you have a USB Mac, you cannot currently connect to the USB port on the SpringBoard. You must use a Serial-to-USB adapter. The adapter must be connected to the serial port on the SpringBoard (the connector with the picture of the computer by it). Select the computer port that your adapter instructions tell you to. If you are using a **Keyspan** adapter, use the **printer port**.
9. Do not change any of the **miscellaneous** settings.
10. Activate the  key on the front of the SpringBoard case.
11. Activate TRANSFER MEMORY MENU.
12. Activate one of the **Save** options to start the saving procedure.

➡ Use the "Load" option when you are loading vocabulary **into** your device. You will be asked if you want to merge the vocabulary you are loading with your current vocabulary or if you want to delete the current vocabulary. If you select **Merge**, the SpringBoard will give precedence to the vocabulary you are loading. If you choose **Delete**, the vocabulary already in the Area will be deleted first, then the new vocabulary loaded.

13. Type in the name of the file you want to use.

➡ To **load** memory, go to the correct drive (or disk) and double click on the file name you want. Click on "Continue".

➡ If you experience a problem, try changing the baud rate for the computer to 57600 in the MTI+ **Baud Rate** menu under **Preferences** in the **Edit** menu on the menu bar. Remember, the baud rate for the SpringBoard and the computer must be the same. Also, make sure the correct computer port has been used and selected.

➡ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the **Shift** key.

Transfer Vocabulary from One SpringBoard to Another

When you plug the MTI-Serial cable into the SERIAL PORT connector on the right side of each SpringBoard, you can transfer *vocabulary* from one SpringBoard to another. If you do not have the MTI-IBM cable, you can order it from PRC.

To Transfer Vocabulary from One SpringBoard to Another:

1. Plug the MTI-Serial-VG-to-IBM-PC cable into the connector with the picture of a computer by it on the side of each SpringBoard. (The ends of the cable are the same so it does not matter which end goes into which connector.)
2. Go to the Toolbox.
3. Activate the TRANSFER MEMORY Menu on each SpringBoard. (Make sure the baud rate is the same for both SpringBoards in the TRANSFER MEMORY MENUs).
4. **On the SpringBoard that contains the vocabulary you want to *transfer***, activate one of the **Save** options to begin the saving procedure.
5. **On the SpringBoard that is going to receive the vocabulary**, activate one of the **Load** options to start the loading procedure.
6. The transfer takes place automatically.

➡ If you want to transfer specific Activities from one device to another, **first** load the Activities into a blank User Area in the device. Then save and load that Area.

➡ **You can also transfer an Area or the entire device contents to the Vantage.** Activate the *Save as Vantage* option in the TRANSFER MEMORY MENU until **On** is visible in the box. Then perform a regular MTI to a computer disk. Then use the MTI to load the file from the computer into the Vantage.

Importing Graphics to the SpringBoard- Windows MTI Only

If you have the Windows MTI, you can import almost any graphic to use in the SpringBoard. You can import photographs that you have scanned, clip art from various web sites, graphics you have drawn using a graphics program, etc.

To import graphics, you need to remember two important rules:

- ➡ Graphics must be in the BMP format.
- ➡ Graphics must be in 256 colors.

If the MTI will not import a graphic, open your graphics program, open the graphic you want to import and check the file format and number of colors for the graphic. If the graphic is not listed as a BMP file in 256 colors, save it using the BMP format and 256 colors.

➡ If you do not have a graphics program on your computer, one is included on the MTI disk. This is a demonstration of Image Viewer/32™ and can only be used for a limited amount of time (30 days).

An Example:

Let's say you have found a graphic on the web that you want to import to the SpringBoard.

1. Point to the graphic with your mouse and click on the **right** mouse button.
2. Click on "Save as" in the menu that pops up.
3. Select the drive and/or file where you want to save the graphic.
4. If the graphic is not in "BMP" form, click on "BMP" in the "Save as type" box. (If "BMP" is not an option in your menu save the graphic as it is.)

2. Change the title of the graphic if you want.

➡ Use up to 8 characters to name a graphic.

➡ Do not name your imported graphic with the same name as an original icon from the SpringBoard default icons. You will not be able to find your graphic once it is imported because the SpringBoard will only acknowledge the original icon.

6. Click on "Save".

You can do one of two things now:

➡ **If you saved the graphic as a "BMP" file:**

You can go ahead and open the MTI and try to import the graphic. See below for instructions. If the graphic was not in 256 colors, the MTI will not let you import it.

➡ **If you could not save the graphic as a BMP file or if you have to check the colors:**

1. Open your graphics program. Open the graphic that you just saved.
2. Open the menu that allows you to select color depth. In a graphics program like *PaintShop Pro*® this is the "Colors" menu. In other programs it might be under a different menu.
3. Set the color to 256. In *PaintShop Pro*®, look at the "Increase Color Depth" and "Decrease Color Depth" options. If "256" is highlighted in either option, select it. If it isn't highlighted in either menu, the graphic was originally done in 256 colors and will transfer.
4. Go to the "File" menu and click on "Save as".
5. Select the drive and/or file where you want to save the graphic.
6. Select "BMP" from the "Save As Type" menu.
7. Change the title of the graphic if you want.
8. Click on "Save".

To Install and Use Image Viewer/32:

1. Put the **MTI CD** in the appropriate drive.
2. In the computer **Start** menu, select **Run**.
3. Type the **drive** you have the disk in (e.g., " a: ")
4. Type "**ivsetup** " (no spaces).
Example: a: ivsetup
5. See the installation screen on your computer.
6. Choose:
Next
Agree
Next
Next
Next
Next
Finish
7. Activate **OK**. Activate **OK** again.
8. Go to the **Start** menu on your computer.
Select **Programs**.
Select **Arcata Pet Software**.
Select **IMGv3.2 Program File**

- Select **OK**.
Open the program.
9. Find the photo or icon you want to use.
 10. Select **Open**.
 11. You should see your picture.
 12. Open the **Image** menu in the menu bar.
 13. Select **Convert To** and choose **256 colors**.
 14. Open the **File** menu in the menu bar.
 15. Select **Save As**.
 16. Select **BMP** and name the graphic.
 17. Select **Save**.


To Import the Graphic to the SpringBoard:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your SpringBoard to the connector labeled **CHARGER** on the top of the SpringBoard. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **IBM cable**, not the SpringBoard's infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled **SERIAL PORT** (picture of a computer by it) on the side of the SpringBoard.
3. Turn the computer On and insert the MTI CD disk into the appropriate drive of your computer. Use "Windows Explorer" to go to the drive that contains the MTI CD or right click on "My Computer" then click on "Explore" to go to the drive that has the disk in it.
4. Double click on the "MTI-EXE" icon. The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., SpringBoard).
6. Go to the SpringBoard's Toolbox and open the **TRANSFER MEMORY MENU**.
7. Activate "Import Icons". If everything is connected correctly, you will see the "Select one or more bitmap files to load" menu on your computer screen.
8. Go to the drive/open the folder where the graphic(s) you want to import is saved.
9. Click on the graphic you want to import. (If you want to import more than one graphic, hold down the **CONTROL** key and click on each graphic that you want to import.)
10. Click on "Open".

11. If you want to change the name of the graphic, do it here. If you want the graphic to have different proportions, remove the "check" from the box.
12. Click "OK".
13. The transfer takes place. When it is finished, click on "OK".

To Check that the Graphic Transferred:

1. Go to the SpringBoard Toolbox.
2. Open the MODIFY PAGE MENU.
3. Select any page, e.g., "Jokes".
4. Activate an unused key.
5. Activate "Change Icon".
6. You see a new key, "IMPORTS" in the Icon Category menu. Activate this key.
7. The graphic you imported should be visible and ready for use.
8. Activate "Cancel" to back your way out of the different MODIFY PAGE menus.
9. If you have graphics that you no longer want in this menu, activate "Delete Icon" and select the icon to be deleted.

 SpringBoard places all imported graphics under the **IMPORTS** key in the Icon Category menu. Activate MODIFY ACTIVITY or MODIFY PAGE to access this menu.

Importing Wav (Sound) Files: Windows MTI Only

In order to transfer wav files to your device, you must first have them stored on your computer. An easy way to find wav files is to type "wav files" into a search engine on your internet browser, then save a few files to a new folder on your computer. ➡ **The files you save must be in PCM format.** To check this: right click on the wav file you want, click on "Properties", then "Summary" to see the format.

To Transfer a Wav File:


1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your Pathfinder to the connector labeled CHARGER on the side of the Pathfinder. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **-IBM cable**, not the SpringBoard's infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the computer port on the side of the SpringBoard.
3. Turn the computer On and insert the MTI CD disk into the appropriate drive. Use "Windows Explorer" to go to the drive that has the MTI CD disk in it or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD disk in it.
4. Double click on "MTI.exe." The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., SpringBoard).
6. Go to your device's Toolbox and select the **STORE CORE** key (you can save wav files using any key or menu which allows you to store vocabulary).
7. Activate the icon or icons you want to store the wav file under, then activate **OK**.
8. Select the **Recorded** option.
9. Select the **Import Sound File** option.
10. On your computer, go to the drive/open the folder where you have your wav files saved.
11. Select the file you want to import and click on "Open".
12. You will see a **Memory Transfer Complete** message box when the file has been transferred. Click on **OK** in the message box. You can select **Listen to Recording** to check that you got the correct file. The wav file is now the recorded message for that icon or icon sequence.
13. Now you can change the icon, the label, the key color, etc., if you wish to.

➡ If you have a previously recorded message assigned to a key or sequence and you import a wav file, the wav file will **replace** the recorded message. Any macros assigned to the key or sequence will remain in place.


Memory Transfers Using a Secure Digital Card Or USB Memory Device

The SpringBoard with Plus features allows you to transfer files from the Springboard to a computer or computer to SpringBoard by using a Secure Digital card or by using a USB memory device (sometimes called a "Keychain" or "Keyfob" device).

Secure Digital Card Transfer

11. Open the Memory door on the left side of the SpringBoard and insert the Secure Digital card in the small slot above the system card. Insert the card face-up (the side with the writing on it should be face up). Push it in until it stays securely. (Push on the end of the card again to remove the card.)
12. Go to the **TOOLBOX** on the SpringBoard and open the **TRANSFER MEMORY MENU**.
13. Change the *MTI Method* option to **SD Card**. If the card is securely in place, you will see a note telling you *Disk Status: READY* and the amount of space available on the card.
 **It may take a few seconds** for the Option name to change in the box after you select an up or down arrow.
14. Now select the memory transfer option you want to use and follow the directions you see on the display screen.
15. You will see a message box telling you when the transfer is completed.

USB Transfer

1. Connect the USB memory device to the **rectangular** USB port on the right side of the SpringBoard case.
2. Go to the **TOOLBOX** and open the **TRANSFER MEMORY MENU**.
3. Change the *MTI Method* option to **USB Disk**. If the USB device is securely in place, you will see a note telling you *Disk Status: READY* and the amount of space available on the card.
 **It may take a few seconds** for the Option name to change in the box after you select an up or down arrow.
4. Now select the memory transfer option you want to use and follow the directions you see on the display screen.
5. You will see a message box telling you when the transfer is completed.

IMPORTANT NOTE

Importing Sound (wav) Files and Saving/Loading Notebooks

You import sound files while you are in storing menus and you save or load Notebooks while you are in the NOTEBOOK MENU. After connecting your memory device, **it is a good idea to go first** to the MEMORY TRANSFER MENU and select either **SD Card** or **USB Disk**. Then begin the storing process for importing sound file or go to the NOTEBOOK MENU to save or load Notebooks.

**MTI for the
Vanguard and the Vanguard for Windows**



Tips for a Successful Windows or DOS Memory Transfer



Plug in the battery charger before you begin a memory transfer..



DO NOT attempt a memory transfer by using the infrared capability. Use the VG cable that came with your MTI disks and connect your Vanguard and computer with the cable.



If you are going to save vocabulary to an IBM disk, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.



If you are going to **download** vocabulary that was programmed for an 8 or 4 location overlay, make sure you are currently using the 8 or 4 overlay in the Vanguard. If you aren't, you must open the CHANGE OVERLAY MENU and switch to the 8 or 4 keyboard and overlay **before** you download the vocabulary.



Before you begin a transfer with this MTI, remove any older versions of the MTI that you may have on your computer



If you are using a Macintosh computer with **AppleTalk** turned On, you must turn it **Off** (inactive) before you can do a memory transfer. Make sure **SerialKeys** is **Off**.



Once the transfer is in progress, do not plug anything into, or unplug anything from, the Vanguard.



When a transfer is in progress, periodically check the computer screen and Vanguard display for any messages or problems.



If you experience a problem, try changing the baud rate to 57600 in the MTI menu bar. Double check that you have selected the computer port that your cable is plugged into.

About Baud Rates

When the Vanguard and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the Vanguard must be the same.**

The default baud rate for the Vanguard II is 57600. You can change the MTI baud rate to match the Vanguard or you change the Vanguard's baud rate to match the MTI program.

To Change the MTI Baud Rate to Match the Vanguard:

If you are using DOS, change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>.

For Windows, change the baud rate in the MTI "Options" menu.

To Change the Vanguard's Baud Rate to Match the MTI:

1. Go to the Toolbox.
2. Activate the TRANSFER MEMORY MENU.
3. Activate the arrows in the "MTI Baud Rate" boxes until you see the baud rate you want to use.
2. Activate OK.

Save Vocabulary on an IBM Computer Using the MTI for Windows[®]

➡ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you begin.

To save memory to an IBM computer running Windows or to load memory back into the Vanguard:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your Vanguard to the connector labeled CHARGER on the top of the Vanguard. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **VG-IBM cable**, not the Vanguard's infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled **SERIAL PORT** on the top of the Vanguard.
3. Turn the computer On and insert the MTI CD disk into the appropriate drive. Use "Windows Explorer" to go to the drive that has the MTI CD disk in it or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD disk in it.
4. Double click on "MTI.exe". The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., Vanguard).
6. Go to the Vanguard's Toolbox and open the TRANSFER MEMORY MENU.
7. Activate the option you want to use (e.g., "Save Entire Device Contents" or "Save One User Area"). If everything is connected correctly, you will see the "Save to Memory File" on your computer screen.
8. On your computer, go to the drive/open the folder where you want to save the memory.
9. Type in a name for the new file, (e.g., VGMEM601). Click on "Save".
10. The transfer could take awhile. When the transfer is finished, click, "OK".

To Restore a File:

1. Select "Load Entire Device" or "Load One User Area" from the TRANSFER MEMORY MENU.
2. On your computer, select the file you want to restore (e.g., VGMEM698).
3. Click on "Restore".

Put the MTI CD on your hard drive and create a desktop shortcut to it:

3. Using "Windows Explorer", go to the drive where you want to place the MTI CD files.

2. Create a new folder and label it "MTI".

3. Go to the drive that contains the MTI CD disk.

4. Right click on "MTI.exe".

5. Left click on "Copy".

6. Go to the drive where you created the new MTI folder. Right click on the folder.

7. Left click on "Paste".

8. Repeat these steps for the "MTI.hlp" file on the MTI CD disk.

➡ Instead of using "Copy" and "Paste", you can move the MTI CD files at once into the new folder if you want. Click on one of the files in the MTI CD disk, then hold down the SHIFT key and click the other MTI CD file. Release the SHIFT key and drag the highlighted files to your new MTI folder.

9. When the files have been moved, double click on the new MTI folder.

10. Right click on "MTI.exe".

11. Left click on "Create Shortcut".

12. The shortcut icon appears. Click on it and drag it to your desktop.

13. From now on you can open the MTI by double-clicking on this icon on your desktop.

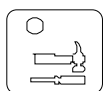
MTI for DOS

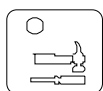
➡ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

➡ You can change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER> Remember the MTI baud rate and the Vanguard baud rate must be the same.

To Save Memory to a Computer Disk or Load it back into the Vanguard:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your Vanguard to the connector labeled CHARGER on the top of the Vanguard. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **MTI-Serial-VG-to-IBM-PC cable**, not the device's infrared capability. Plug the one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled **SERIAL PORT** on the top of the Vanguard.
3. Turn the computer On and insert the MTI CD into the appropriate drive. Switch to the drive that has the disk in it if necessary.
4. Type "DOSMTI.exe" and press <RETURN>.
5. Type the letter of the device you are using: in this case type "V".



6. Press the  key on the right front of the Vanguard case. You see the Toolbox screen.
 7. Activate the TRANSFER MEMORY MENU in the Toolbox.
 8. Activate "Save Entire Device Contents" or "Save One User Area" to start the saving process.
- ➡ Use "Load Entire Device" or "Load One User Area" when you are loading vocabulary **into** your Vanguard. Use "Merge" when you are downloading a MAPMate.
9. On the computer, type in the name of the file you want to save the memory to (e.g., VGMEM1). Press F1 to complete the transfer.
- ➡ To **load** memory, select the file on the computer that you want to load the memory from.



Tips for a Successful MTI+ Memory Transfer (Macintosh)



Plug in the battery charger before you begin a memory transfer.



DO NOT attempt a memory transfer by using the infrared capability. Use the cable that came with your MTI+ disk and connect your Vanguard and computer with the cable.

➡ The MTI+ does not work well on the following Macs running a system software version that is older than 7.5.3: Power Mac 6100, 7100 and 8100; Centris/Quadra 660av and 840av. You can update software from the following internet address: <http://asu.info.apple.com/>

➡ If you are using a Macintosh that only has a USB port, you will have to purchase a "USB-to-Serial" adapter. This will allow your MTI-MAC cable to work with the computer. PRC has tested an adapter made by *Keyspan*. Go to the Keyspan web page or do a computer search on "USB to Serial Adapters"

➡ If you want to put the vocabulary on floppy disks, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➡ If you have **AppleTalk** turned On, you must turn it **Off** (inactive) before you can do a memory transfer. Make sure **SerialKeys** is turned **Off**.

➡ Once the transfer is in progress, do not plug anything into, or unplug anything from, the Vanguard.

➡ When a transfer is in progress, periodically check the computer screen and Vanguard display for any messages or problems.

➡ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ **Preferences** menu (inside the **Edit** menu). You may have to change the **port setting** on your computer. If you **are not** using a USB port, make sure the computer port you selected is the one your cable is plugged into, i.e., Modem or Printer. If you are using the **USB** port, the Keyspan adapter requires you to set the port to **Printer**. If you are using a different adapter, the information that came with the adapter should tell which computer port to set.

➡ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the **Shift** key.

About Baud Rates

When the Vanguard and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for the MTI program and the baud rate for the Vanguard must be the same.**

The default baud rate for the Vanguard is 57600. You can change the MTI baud rate to match the Vanguard or you change the Vanguard's baud rate to match the MTI.

To Change the MTI Baud Rate to Match the Vanguard:

For the **MTI+**, click the **Edit** menu on the top of the computer display screen. Click **Preferences**. Click on the **Baud Rate** menu and select the rate you want.

To Change the Vanguard's Baud Rate to Match the MTI:

Go to the Toolbox.

Activate the TRANSFER MEMORY MENU.


Activate the arrows in the "MTI Baud Rate" boxes until you see the baud rate you want to use.

Activate OK.

Using the MTI+ for Macs

1. All memory transfers should be made with the **battery charger plugged in**. Plug the battery charger that came with your Vanguard into the CHARGER connector on the Vanguard. Plug the other end into a wall outlet.
2. **All memory transfers should be made using the MTI+ cable** that came with your MTI+. If you are using an older Mac, plug one end of the cable into either the Printer or Modem connector on the back of the computer. Plug the other end into the connector that has the picture of a computer above it on the Vanguard. If you are using a USB Mac, plug one end of the cable into the USB port on the computer. Plug the adapter and the other end of the cable into the connector that has the picture of a computer above it on the Vanguard.
3. Turn the computer On and insert the MTI CD. Double-click on the MTI+ icon to open the program.
4. **Copy the MTI+ to your Hard Drive for Better Performance:**
Click and hold on the folder labeled **MTI+**. Drag the folder to a location of your choice on your hard drive.
Close any open programs or applications. Then, double-click on the MTI+ folder on your hard drive to open the program.

Making a Transfer

5. Open the EDIT menu at the top of your computer screen and click on *Preferences*.
6. Make sure **Vanguard** is selected in the **Device** menu.
7. Set the **baud rate** to the rate you want to use. The rate that is visible to begin with is the default baud rate for your selected device. The MTI baud rate and the Vanguard baud rate must be the same.
8. Select a **computer port**. If you are using an older Mac, select the port you connected your cable to (either Printer or Modem). If you are using a USB Mac, the instructions that came with your adapter should tell you which port to select. The Keyspan adapter requires you to use the **Printer** port.
9. Do not change any of the **miscellaneous** settings.
10. Activate the  key on the front of the Vanguard case.
11. Activate TRANSFER MEMORY MENU.

12. Activate "Save Entire Device Contents" or "Save One User Area" to start the saving procedure.

➤ Use the "Load" option when you are loading vocabulary **into** your device. Use "Merge" if you are adding new vocabulary to your current vocabulary.

13. Type in the name of the file you want to use.

➤ To **load** memory, go to the correct drive (or disk) and double click on the file name you want. Click on "Continue".

➤ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ *Preferences* menu (inside the **Edit** menu). You may have to change the **port setting** on your computer. If you **are not** using a USB port, make sure the computer port you selected is the one your cable is plugged into, i.e., Modem or Printer. If you are using the **USB** port, the Keyspan adapter requires you to set the port to **Printer**. If you are using a different adapter, the information that came with the adapter should tell which computer port to set.

➤ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the **Shift** key.

Transfer Vocabulary from One Vanguard to Another

When you plug the MTI-Serial-VG-to-IBM-PC cable into the SERIAL PORT connector on the top of each Vanguard, you can transfer *vocabulary* from one Vanguard to another. If you do not have the MTI-IBM cable, you can order it from PRC.

To Transfer Vocabulary from One Vanguard to Another:

1. Plug the MTI-Serial-VG-to-IBM-PC cable into the SERIAL PORT connector on the back of each Vanguard. (The ends of the cable are the same so it does not matter which end goes into which connector.)
2. Go to the Toolbox.
3. Activate the TRANSFER MEMORY Menu on each Vanguard. (Make sure the baud rate is the same for both Vanguards in the OUTPUT MENU).
4. **On the Vanguard that contains the vocabulary you want to *transfer***, activate "SAVE ONE USER AREA" or "SAVE ENTIRE DEVICE CONTENTS" to begin the saving procedure.
5. **On the Vanguard that is going to receive the vocabulary**, activate LOAD ONE USER AREA or "LOAD ENTIRE DEVICE CONTENTS" to start the loading procedure.
6. The transfer takes place automatically.

➡ If you want to transfer specific Pages or Activities from one device to another, **first** load the Pages/Activities into a blank User Area in the device. Then save and load that Area.

Importing Graphics to the Vanguard- Windows MTI Only

If you have the Windows MTI, you can import almost any graphic to use in the Vanguard. You can import photographs that you have scanned, clip art from various web sites, graphics you have drawn using a graphics program, etc.

To import graphics, you need to remember two important rules:

- ➡ Graphics must be in the BMP format.
- ➡ Graphics must be in 256 colors.

If the MTI will not import a graphic, open your graphics program, open the graphic you want to import and check the file format and number of colors for the graphic. If the graphic is not listed as a BMP file in 256 colors, save it using the BMP format and 256 colors.

➡ If you do not have a graphics program on your computer, one is included on the MTI disk. This is a demonstration of Image Viewer/32™ and can only be used for a limited amount of time (30 days).

An Example:

Let's say you have found a graphic on the web that you want to import to the Vanguard.

14. Point to the graphic with your mouse and click on the **right** mouse button.
2. Click on "Save as" in the menu that pops up.
3. Select the drive and/or file where you want to save the graphic.
4. If the graphic is not in "BMP" form, click on "BMP" in the "Save as type" box. (If "BMP" is not an option in your menu save the graphic as it is.)
3. Change the title of the graphic if you want.

- ➡ Use up to 8 characters to name a graphic.
- ➡ Do not name your imported graphic with the same name as an original icon from the Vanguard default icons. You will not be able to find your graphic once it is imported because the Vanguard will only acknowledge the original icon.

6. Click on "Save".

You can do one of two things now:

➡ **If you saved the graphic as a "BMP" file:**

You can go ahead and open the MTI and try to import the graphic. See below for instructions. If the graphic was not in 256 colors, the MTI will not let you import it.

➤ **If you could not save the graphic as a BMP file or if you have to check the colors:**

1. Open your graphics program. Open the graphic that you just saved.
15. Open the menu that allows you to select color depth. In a graphics program like *PaintShop Pro*® this is the "Colors" menu. In other programs it might be under a different menu.
16. Set the color to 256. In *PaintShop Pro*®, look at the "Increase Color Depth" and "Decrease Color Depth" options. If "256" is highlighted in either option, select it. If it isn't highlighted in either menu, the graphic was originally done in 256 colors and will transfer.
17. Go to the "File" menu and click on "Save as".
5. Select the drive and/or file where you want to save the graphic.
6. Select "BMP" from the "Save As Type" menu.
7. Change the title of the graphic if you want.
8. Click on "Save".

To Install and Use Image Viewer/32:

1. Put the **MTI** in the appropriate drive.
2. In the computer **Start** menu, select **Run**.
3. Type the **drive** you have the MTI in (e.g., " d: ")
4. Type " **ivsetup** " (no spaces).
Example: a: ivsetup
5. See the installation screen on your computer.
6. Choose:
 - Next
 - Agree
 - Next
 - Next
 - Next
 - Next
 - Next
 - Finish
7. Activate **OK**. Activate **OK** again.
8. Go to the **Start** menu on your computer.
 - Select **Programs**.
 - Select **Arcata Pet Software**.
 - Select **IMGv3.2 Program File**
 - Select **OK**.

Open the program.

9. Find the photo or icon you want to use.
10. Select **Open**.
11. You should see your picture.
12. Open the **Image** menu in the menu bar.
13. Select **Convert To** and choose **256 colors**.
14. Open the **File** menu in the menu bar.
15. Select **Save As**.
16. Select **BMP** and name the graphic.
17. Select **Save**.


To Import the Graphic to the Vanguard:

1. All memory transfers should be made with the **battery charger plugged in**. Connect one end of the battery charger that came with your Vanguard to the connector labeled CHARGER on the top of the Vanguard. Plug the other end into a wall outlet.
2. All memory transfers should be made with the **VG-IBM cable**, not the Vanguard's infrared capability. Plug one end of this cable into Serial Port 1 on your computer. Plug the other end into the connector labeled **SERIAL PORT** on the top of the Vanguard.
3. Turn the computer On. If you have stored the MTI on your hard drive, double click on the MTI icon on your computer screen. Otherwise, insert the MTI CD into the appropriate drive of your computer. Use "Windows Explorer" to go to the drive that contains the MTI CD or right click on "My Computer" then click on "Explore" to go to the drive that has the disk in it.
4. Double click on the "MTI-EXE" icon. The MTI screen is visible on your computer monitor.
5. In the MTI "Options" menu, click on the device you are using (e.g., Vanguard).
6. Go to the Vanguard's Toolbox and open the TRANSFER MEMORY MENU.
7. Activate "Import Icons". If everything is connected correctly, you will see the "Select one or more bitmap files to load" menu on your computer screen.
8. Go to the drive/open the folder where the graphic(s) you want to import is saved.
9. Click on the graphic you want to import. (If you want to import more than one graphic, hold down the CONTROL key and click on each graphic that you want to import.)
10. Click on "Open".

11. If you want to change the name of the graphic, do it here. If you want the graphic to have different proportions, remove the "check" from the box.
12. Click "OK".
13. The transfer takes place. When it is finished, click on "OK".

To Check that the Graphic Transferred:

5. Go to the Vanguard Toolbox.
2. Open the MODIFY PAGE MENU.
3. Select any page, e.g., "Jokes".
4. Activate an unused key.
5. Activate "Change Icon".
6. You see a new key, "IMPORTS" in the Icon Category menu. Activate this key.
7. The graphic you imported should be visible and ready for use.
8. Activate "Cancel" to back your way out of the different MODIFY PAGE menus.
9. If you have graphics that you no longer want in this menu, activate "Delete Icon" and select the icon to be deleted.

 Vanguard places all imported graphics under the **IMPORTS** key in the Icon Category menu. Activate MODIFY ACTIVITY or MODIFY PAGE to access this menu.

MTI for AlphaTalker



Tips for a Successful Windows or DOS Memory Transfer



Plug the battery charger in **before** you begin a transfer.



Remove any previous versions of MTI software you may have on your computer **before** installing this MTI version.



Once the transfer is in progress, **do not** plug anything into, or unplug anything from, the AlphaTalker.



Digitized speech creates large vocabulary files. Have 2 or 3 formatted disks ready to save your vocabulary on **before** you begin the transfer.



If you are using MTI-IBM-DOS with Windows, depending on the speed of your computer, the MTI may not run. If this happens, restart your computer and use DOS mode to run the MTI, or use the MTI CD.

About Baud Rates

When the AlphaTalker and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the AlphaTalker must be the same.**

The default baud rate for the AlphaTalker is 57600. You can change the MTI baud rate to match the AlphaTalker or you change the AlphaTalker's baud rate to match the computer's.

To Change the MTI Baud Rate to Match the AlphaTalker:

If you are using DOS, change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>.

If you are using Windows, change the baud rate in the MTI "Options" menu.

To Change the AlphaTalker's Baud Rate to Match the MTI:

1. Remove the keyguard and overlay. Press SELECT in the Control Panel and step to TOOLS. Press the ON/OFF side to turn the Toolbox On.
2. Press SET BAUD RATE in the Toolbox. All the LEDs in the Toolbox go out *except the current baud rate*. This also flashes.
3. Notice the numbers in the top right corners of keys C-4, D-4, E-4 and F-4. Each of these numbers is a baud rate. If you want the current baud rate (where the LED is lit) just activate that key. If you want a different baud rate, activate the key with the number on it that you want.
4. When you select a baud rate, that LED flashes for a few seconds. When the light goes out, you can continue to use the Toolbox or you can go to the Control Panel and turn the Toolbox Off by pressing SELECT and then the ON/OFF side

Save Vocabulary Using the MTI for Windows®

➤ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.


Using the MTI:

1. Remove the keyguard and overlay and open the Toolbox.
2. All memory transfers should be made with the battery charger plugged in. Connect one end of the battery charger that came with your AlphaTalker to the connector on the side of the case. Plug the other end of the battery charger into a working wall outlet.
3. Plug the larger rectangular end of the cable that came with your MTI program into Serial Port 1 on your computer. Plug the small round end into the connector labeled **RS232** on the top of the AlphaTalker case.
4. Turn the computer on. Insert the MTI CD disk into the correct drive of your computer. Use "Windows Explorer" or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD in it.
5. Double click on "MTI.exe". The MTI screen is visible on your computer monitor.
6. In the MTI "Options" menu, click on the name of the device you are using (i.e., 'AlphaTalker'). (You can also change computer serial ports here and select a baud rate if you need to.)
7. To save memory to the computer: Go to the AlphaTalker Toolbox and press the SAVE MEMORY TO DISK key. If every thing is connected properly, you will see "Save Memory to File" on your computer screen.
8. On your computer, go to the drive/folder where you want to save the AlphaTalker 's memory.
9. Type in a name for the new file (e.g., ATMEM1). Click on "Save". The transfer takes place automatically.

To Restore Memory to the AlphaTalker:

1. Select the LOAD MEMORY FROM DISK key in the AlphaTalker's Toolbox.
2. On your computer, select the AlphaTalker file you want to restore (e.g., ATMEM1).
3. Click on "Restore".

Put the MTI CD on your hard drive and create a shortcut to it:

1. Put the MTI CD disk in the correct drive of your computer.
2. Use the "Windows Explorer" to go to the drive where you want to place the MTI CD.
3. Create a new folder and label it " MTI".
4. Go to the drive that contains the MTI CD disk.
5. Right click on "MTI.exe".
6. Left click on "Copy".
7. Go to the drive where you created the MTI folder. Right click on the folder.
8. Left click on "Paste".
9. Repeat these steps for the "MTI.hlp" file. Add it to the new MTI folder.
 Instead of using "Copy" and "Paste", you can move the original MTI CD files at once into the new MTI CD folder. Click on one of the files on the MTI CD disk. Then, hold down the <SHIFT> key and click on the other file. Release the <SHIFT> key and drag the highlighted files to your new MTI folder.
10. When the files have been moved to the new folder, double click on the new MTI folder.
11. Right click on "MTI.exe".
12. Left click on "Create Shortcut".
13. The shortcut icon appears. Click on it and drag it to your desktop.
14. From now on you can open the MTI by double-clicking on this icon on your desktop.

MTI for DOS

➤ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

Using the MTI

1. Remove the keyguard and overlay from the AlphaTalker.
2. All memory transfers should be made with the battery charger plugged in. Connect one end of the battery charger that came with your AlphaTalker to the connector labeled BATTERY CHARGER on the back of the AlphaTalker. Plug the other end into a wall outlet.
3. Plug the large end of the cable that came with your MTI program into Serial Port 1 on your computer. Plug the small round end into the connector labeled RS-232 on the back of the AlphaTalker.
4. Turn the computer On and insert the MTI CD into the appropriate drive. Switch to the drive that has the disk in it if necessary.
5. Type "DOSMTI.exe" and press <RETURN>.
6. Type the letter of the device you are using: in this case type "A".
7. Open the AlphaTalker's Toolbox in the Control Panel by pressing OPTION SELECT. Step to TOOLS. Press OPTION ON/OFF.
8. In the Toolbox, press SAVE MEMORY or LOAD MEMORY. Close the Toolbox by pressing OPTION SELECT and then OPTION ON/OFF. The TOOLS light goes out.
9. You are not allowed to **save** memory onto the MTI CD so choose a different disk or change to another drive to save memory to. On the computer, type in the name of the file you want to save the memory to.
- 9a. To **load** memory, select the file you want to load the memory from.
10. Read the warning and then press F1 to continue.

➤ **Note:** If you experience a problem, you can change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example:
C:/dosmti 9600 <ENTER> Also, make sure you have the correct computer port selected.



Tips for a Successful MAC+ Memory Transfer



Plug in the battery charger before you begin a memory transfer.

➤ The MTI+ does not work well on the following Macs running a system software version that is older than 7.5.3: Power Mac 6100, 7100 and 8100; Centris/Quadra 660av and 840av. You can update software from the following internet address: <http://asu.info.apple.com/>

➤ If you are using a Macintosh that only has a USB port, you will have to purchase a "USB-to-Serial" adapter. PRC has tested the adapter made by *Keyspan*. Check the Keyspan web page or do a computer search for "USB to serial adapters".

➤ If you want to put the vocabulary on floppy disks, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➤ If you have **AppleTalk** turned On, you must turn it **Off** (inactive) before you can do a memory transfer. (Restart your computer. As it restarts, hold the <SHIFT> key down to run off any extensions.) Make sure SerialKeys is **Off**.

➤ Once the transfer is in progress, do not plug anything into, or unplug anything from, the AlphaTalker.

➤ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ *Preferences* menu (inside the **Edit** menu). You may have to change the **port setting** on your computer. If you **are not** using a USB port, make sure the computer port you selected is the one your cable is plugged into, i.e., Modem or Printer. If you are using the **USB** port, the Keyspan adapter requires you to set the port to **Printer**. If you are using a different adapter, the information that came with the adapter should tell which computer port to set.

➤ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the **Shift** key.

About Baud Rates

When the AlphaTalker and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the AlphaTalker must be the same.**

The default baud rate for the AlphaTalker is 57600. You can change the MTI+ baud rate to match the AlphaTalker or you can change the AlphaTalker's baud rate to match the MTI+ program.

To Change the MTI+ Baud Rate to Match the AlphaTalker:

For the MTI+, click the **Edit** menu on the top of the computer display screen. Click **Preferences**. Click on the **Baud Rate** menu and select the rate you want.

To Change the AlphaTalker's Baud Rate to Match the MTI+:

1. Remove the keyguard and overlay. Press SELECT in the Control Panel and step to TOOLS. Press the ON/OFF side to turn the Toolbox On.
2. Press SET BAUD RATE in the Toolbox. All the LEDs in the Toolbox go out *except the current baud rate*. This also flashes.
3. Notice the numbers in the top right corners of keys C-4, D-4, E-4 and F-4. Each of these numbers is a baud rate. If you want the current baud rate (where the LED is lit) just activate that key. If you want a different baud rate, activate the key with the number on it that you want.
4. When you select a baud rate, the LED flashes for a few seconds. When the light goes out, you can continue to use the Toolbox or you can go to the Control Panel and turn the Toolbox Off by pressing SELECT and then the ON/OFF side

Using the MTI+ for Macs

1. All memory transfers should be made with the **battery charger plugged in**. Plug the battery charger that came with your Pathfinder into the CHARGER connector on the side of the Pathfinder. Plug the other end into a wall outlet.
2. All memory transfers should be made using the **cable** that came with your MTI+. If you are using an **older Mac**, plug one end of the cable into the connector on the back of the computer that has the picture of the telephone handset above it or the picture of the printer. Plug the other end into the RS232 connector on the back of the AlphaTalker. If you are using a **USB Mac**, plug the cable into the USB port. Plug the adapter and other end of the cable into the RS232 port on the AlphaTalker.
3. Turn the computer On and insert the MTI CD. Double-click on the icon to open the program.
4. **Copy the MTI+ to your Hard Drive for Better Performance:**
Click and hold on the folder labeled **MTI+**. Drag the folder to a location of your choice on your hard drive.
Close any open programs or applications. Then, double-click on the MTI+ folder on your hard drive to open the program.

Making a Transfer

5. Open the EDIT menu at the top of your computer screen and click on *Preferences*.
6. Make sure **AlphaTalker** is selected in the **Device** menu.
7. Set the baud rate to the rate you want to use. The rate that is visible to begin with is the default baud rate for your selected device. This baud rate **must be the same** as the baud rate in the AlphaTalker.
8. **Select a Computer Port:** If you are using an older Mac, select the computer port you plugged your cable into, either Printer or Modem. If you are using a USB Mac, select the computer port that your adapter instructions tell you to use. If you are using a Keyspan adapter, you must select the **Printer** port.
9. Do not change any of the **Miscellaneous** settings.
10. Open the AlphaTalker's Toolbox in the Control Panel.
11. In the Toolbox, press SAVE MEMORY or LOAD MEMORY.
To Save Memory: select the disk or drive where you want to save the memory. Type in the file name you want to use. Close the Toolbox.

To Load Memory: Go to the correct disk or drive where you saved the memory. Double click on the name of the file you want to load.

➡ Use the "Load" option when you are loading vocabulary **into** your device. Use "Merge" if you are adding new vocabulary to your current vocabulary.

➡ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ *Preferences* menu (inside the **Edit** menu). You may have to change the **port setting** on your computer. If you **are not** using a USB port, make sure the computer port you selected is the one your cable is plugged into, i.e., Modem or Printer. If you are using the **USB** port, the Keyspan adapter requires you to set the port to **Printer**. If you are using a different adapter, the information that came with the adapter should tell which computer port to set.

➡ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the **Shift** key.

Transfer Vocabulary from One AlphaTalker to Another

By plugging the MTI+ (Macintosh) cable in to the RS-232 ports on the top of each AlphaTalker, you can transfer *vocabulary* (not system information) from one AlphaTalker to another. If you do not have the MTI+ cable, you can order it from PRC.

Using the MTI:

1. Plug the MTI+ cable into the RS-232 connectors on the top panel of each AlphaTalker. (The ends of the cable are the same so it doesn't matter which end goes into which connector.)
2. Remove the keyguard and overlay from each AlphaTalker. Go to the Control Panel on each AlphaTalker. Step to TOOLS, then activate OPTION ON/OFF to open the Toolbox.
3. Activate SAVE MEMORY in the Toolbox of the AlphaTalker that has the **vocabulary you want to transfer**.
4. Activate LOAD MEMORY in the Toolbox of the AlphaTalker you want to **load the saved memory into**.
5. The transfer takes place automatically. Lights will blink on the AlphaTalker that is transferring its memory. When the transfer is finished the lights will stop blinking.

AlphaTalker Keyboard Emulation

Keyboard emulation allows you to set up the AlphaTalker keyboard so that it functions like a computer keyboard and therefore allows you to use a computer. It also lets you use the AlphaTalker to run environmental control units (ECU's) like the Director or the Control-1. You must have the MTI software and a keyboard emulator like the T-TAM or an ECU in order to use Keyboard Emulation. Once you have created an overlay you will not have to use the MTI unless you want to go back and modify the original overlay or create a new one.

➡ **If you have an Apple computer** you have to create the emulation overlay on a Macintosh or IBM computer. Once the overlay is created, you can load it into the AlphaTalker and connect the T-TAM, the Apple and the AlphaTalker and use the emulation overlay.

➡ **When you are using Keyboard emulation you cannot store or speak messages.** You must turn the Keyboard Emulation tool Off in order to use the AlphaTalker for speaking.

➡ Before you begin to create an emulation overlay, know what the computer (or ECU) commands are that you want to put on the overlay. See Appendix A for a list of Apple/Macintosh and IBM computer key commands.

➡ **Save the vocabulary in the AlphaTalker before you begin.**

➡ **Make sure the baud rates between the AlphaTalker and the emulation program are the same.**



Tips for a Successful Emulation Transfer

- **Always** save the AlphaTalker's vocabulary *first*.
- Once you save the emulation overlay, you must **load** it into the AlphaTalker. Use the MTI.
- When you are ready to use the emulation overlay, set the AlphaTalker's baud rate to the same rate that your emulator uses.
- If you are using a Macintosh that only has a USB port, you will have to purchase a "USB-to-Serial" adapter. This will allow your MTI-MAC cable to work with the computer.

➤ **About Entering Emulation Commands:**

You must type the overlay commands in the language that your keyboard emulator or ECU understands. For example, all keyboard emulator manuals tell you that to program an "UP" key, you must press the ESCAPE key on the computer, type UP and then type a period(.).

The DIRECTOR manual tells you that you must start each command with NULL ESC (NULL is represented by activating the CTRL and @ keys: CTRL@ESC) and then continue with specific instructions until you end with a period. (On those Macintosh's which do not have a Control key you must use the Command key.

For more information about entering commands correctly, read the manual that came with your emulator or ECU. ➤ See Appendix A in this manual for a list of computer key commands that can be used for emulation.

➤ **8 and 4 Location Emulation Overlays:**

With an 8 or 4 Location emulation overlay, you must program your instructions into the **upper left corner key** in each block of keys.

➤ When you are using keyboard emulation you cannot speak or store messages. You must turn KEYBOARD EMULATION Off in the Toolbox before you can speak or store. You can assign the KEYBOARD EMULATION tool to a key on your overlay if you would like to have it handy.

➤ **Remember:** if you assign KEYBOARD EMULATION to a key, you cannot assign an emulation command to that key when you are creating your emulation.

Keyboard Emulation-IBM

Remove the keyguard and overlay. Use the MTI program, **to save the AlphaTalker's vocabulary first** (see the *MTI-IBM* Instructions) then:

1. Press F9 on the computer keyboard (DOS, 3.1) or select Keyboard Emulation (Windows).
2. Select the file you want to use for emulation (we suggest you use the just saved vocabulary file).
3. Create the emulation overlay. Select squares on the computer screen by pressing the arrow keys on the computer keyboard. Type the directions you want the key to contain. You may type up to 63 characters per key, but only a few of them will show on the overlay squares.
4. When you have finished, you can exit the program without saving the emulation (practice) or you can save the emulation by pressing F5.
5. **Load the saved emulation back into the AlphaTalker.** Turn the Toolbox On. Activate LOAD MEMORY (F4) and follow the directions on your computer screen.
6. When the file is loaded, exit the MTI program, disconnect the MTI cable and connect your keyboard emulator or ECU to the AlphaTalker and the computer (see the emulator manual).
7. Turn the Toolbox On in the Control Panel. **Activate BAUD RATE in the Toolbox and then the baud rate that your emulator uses (usually 300 or 1200).** Turn KEYBOARD EMULATION (D4) ON. Make sure the program you want to use is running on your computer or in the case of an ECU, that the ECU is turned On.

➡ With an 8 or 4 location emulation overlay you must program instructions into the **upper left corner key** in each block of keys.

Keyboard Emulation—Macintosh

Remove the keyguard and overlay. Boot up the MTI+ program. **Always save the AlphaTalker vocabulary first** (see the *MTI+ instructions* earlier in this chapter).

1. Click on "AT Keyboard Emulation" in the **Tools** menu at the top of the program.
2. Click on the file you want to use (use the vocabulary file you just saved).
3. Select keys on the computer screen by clicking on them. Type the directions you want the key to contain. You may type up to 63 characters per key, but only a few of them will show on the overlay squares.

➤ **HEPFUL HINT:** The pop-up menu on the left of the MTI+ emulation screen gives you various computer control characters. You can select the ESC character from the list and then type in the command you want to use. This saves you from activating CTRL[before every command.

4. When you have finished, you can exit the program without saving the emulation ("Cancel") or you can save the emulation by clicking on "Save".
5. Once you save the emulation, **load it back into the AlphaTalker**. Turn the Toolbox On. Activate LOAD MEMORY (F4) and follow the directions on your computer screen.
6. When the file is loaded, exit the MTI+ program, disconnect the MTI+ cable and connect your keyboard emulator or ECU to the AlphaTalker and the computer (see the emulator manual).
7. Turn the Toolbox On in the Control Panel. **Activate BAUD RATE (C4) in the Toolbox and then the baud rate that your emulator uses (usually 300 or 1200)**. Turn KEYBOARD EMULATION (D4) ON. Make sure the program you want to use is running on your computer or in the case of an ECU, that the ECU is turned On.

➤ With an 8 or 4 location emulation overlay you must program instructions into the **upper left corner key** in each block of keys.

Copy the ASSIGN KEYBOARD EMULATION Key from the Toolbox to Your Overlay

If you would like to have the KEYBOARD EMULATION tool on your regular AlphaTalker overlay, you can copy the ASSIGN KEYBOARD EMULATION key (G4) from the Toolbox to your overlay. Once this key is copied on your overlay, you use it the same way you use the KEYBOARD EMULATION tool in the Toolbox. That is, you just activate the key on your overlay where you copied ASSIGN KEYBOARD EMULATION to put the AlphaTalker into keyboard emulation. To exit keyboard emulation you just activate the key again.

When you copy a key from the Toolbox to your overlay you can still use the key in the Toolbox if you want, as well as the key on your overlay.

To Copy the ASSIGN KEYBOARD EMULATION Key to Your Overlay:

1. Press SELECT in the Control Panel and step to TOOLS. Press the ON/OFF side to open the Toolbox.
2. Press STORE in the Control Panel.
3. The three keys in the Toolbox that you can copy to your overlay light up. Press the ASSIGN KEYBOARD EMULATION key.
4. Press the key on your overlay where you want to put ASSIGN KEYBOARD EMULATION. (Icon Prediction allows you to see where you have blank keys. No LEDs will be lit at the blank keys.)
5. You hear three quick beeps. This tells you that the keyboard emulation key has been copied onto your overlay.
6. Turn the Toolbox Off in the Control Panel. (Press SELECT. When the TOOLS light flashes, press the ON/OFF side.)

MTI for DeltaTalker



Tips for a Successful Windows or DOS Memory Transfer



Plug the battery charger in **before** you begin a transfer.



If you have the infrared software up and running on your DeltaTalker:

DO NOT attempt a memory transfer by using the infrared capability. Use the MTI cable that is supplied with your MTI CD and connect your DeltaTalker and computer with the cable.



If you are going to save vocabulary to an IBM disk, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.



The following is an especially important hint!

If you are going to **download** vocabulary that was programmed for an 8 or 32 location overlay, make sure you are currently using the 8 or 32 set-up in the DeltaTalker. If you aren't, you must open the CHANGE OVERLAY menu and switch to the 8 or 32 keyboard and overlay **before** you download the vocabulary.



If you are using DOS with Windows, depending on the speed of your computer, the MTI may not run. If this happens, restart your computer and use DOS mode to run the MTI, or use the MTI CD.



Once the transfer is in progress, do not plug anything into, or unplug anything from, the DeltaTalker.



When a transfer is in progress, periodically check the computer screen and DeltaTalker display for any messages or problems.

About Baud Rates

When the DeltaTalker and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the DeltaTalker must be the same.**

The default baud rate for the DeltaTalker is 57600. You can change the MTI baud rate to match the DeltaTalker or you change the DeltaTalker's baud rate to match the computer's.

To Change the MTI Baud Rate to Match the DeltaTalker:

If you are using DOS, change the MTI program's baud rate when you boot up the program. Type "dosmti <space>the new baud rate<enter>". For example: C:/dosmti 9600 <ENTER>.

If you are using Windows, change the baud rate in the MTI "Options" menu.

To Change the DeltaTalker's Baud Rate to Match the MTI:

1. Turn the Toolbox On.
2. Activate the TRANSF. MEMORY menu.
3. Activate NEXT MENU ITEM to step to "MTI Baud Rate = >57600<.
4. Activate CHANGE >ITEM< until you step to the baud rate you want to use.
5. Activate EXIT.

Save Vocabulary Using the MTI for Windows[®]

➡ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

Using the MTI CD:

1. Remove the keyguard and overlay and open the Toolbox.
2. All memory transfers should be made with the battery charger plugged in. Connect one end of the battery charger that came with your DeltaTalker to the connector on the top of the case. Plug the other end of the battery charger into a working wall outlet.
3. Plug the larger rectangular end of the cable that came with your MTI program into Serial Port 1 on your computer. Plug the small round end into the RS232 connector on the top of the DeltaTalker case.
4. Turn the computer on. Insert the MTI CD disk into the appropriate drive. Use "Windows Explorer" or right click on "My Computer" and click on "Explore" to go to the drive that has the MTI CD in it.
5. Double click on "MTI.exe". The MTI screen is visible on your computer monitor.
6. In the MTI "Options" menu, click on the name of the device you are using (i.e., 'DeltaTalker'). You can also change computer serial ports here and select a baud rate if you need to.
7. To save memory to the computer: Go to the DeltaTalker Toolbox. Activate TRANSFER MEMORY MENU. Activate NEXT MENU ITEM until you see SAVE MEMORY TO DISK. Activate CHANGE >ITEM<. If every thing is connected properly, you will see "Save Memory to File" on your computer screen.
8. On your computer, go to the drive/folder where you want to save the DeltaTalker's memory.
9. Type in a name for the new file (e.g., DTMEM701). Click on "Save". The transfer takes place automatically.

To Restore Memory to the DeltaTalker:

3. Select the LOAD MEMORY FROM DISK from the TRANSFER MEMORY MENU in the DeltaTalker Toolbox.
4. On your computer, select the DeltaTalker file you want to restore (e.g., DTMEM701).
5. Click on "Restore".

Put the MTI CD on your hard drive and create a shortcut to it:

1. Make sure the MTI CD is in the correct drive of your computer.
2. Using the "Windows Explorer", go to the drive where you want to place the MTI CD.
3. Create a new folder and label it, " MTI".
4. Go to the drive that contains the MTI CD disk.
5. Right click on "MTI.exe".
6. Left click on "Copy".
7. Go to the drive where you created the "MTI" folder. Right click on the folder.
8. Left click on "Paste".
9. Repeat these steps for "MTI.hlp" file. Add it to the new "MTI" folder.

➡ Instead of using "Copy" and "Paste", you can move the MTI CD files at once into the new "MTI" folder. Click on one of the files on the MTI CD disk. Then, hold down the <SHIFT> key and click on the other file. Release the <SHIFT> key and drag the highlighted files to your new "MTI" folder.

10. When the files have been moved to the new folder, double click on the new "MTI" folder.
11. Right click on "MTI.exe".
12. Left click on "Create Shortcut".
13. The shortcut icon appears. Click on it and drag it to your desktop.
14. From now on you can open the MTI by double-clicking on this icon on your desktop.

MTI for DOS

☞ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

To Transfer Memory:

1. Remove the keyguard and overlay from the DeltaTalker.
2. All memory transfers should be made with the battery charger plugged in. All memory transfers should be made with the cable, not the DeltaTalker infrared capability. Connect one end of the battery charger that came with your DeltaTalker to the connector labeled BATTERY CHARGER on the back of the DeltaTalker. Plug the other end into a wall outlet.
3. Plug the large end of the cable that came with your MTI program into Serial Port 1 on your computer. Plug the small round end into the connector labeled RS-232 on the back of the DeltaTalker.
4. Turn the computer On and insert the MTI CD disk into the appropriate drive. Switch to the drive that has the disk in it if necessary.
5. Type "DOSMTI.exe" and press <RETURN>.
6. Type the letter of the device you are using: in this case type "E".
7. Turn on the DeltaTalker's **Toolbox**.
8. Activate the TRANSF. MEMORY menu in the Toolbox.
9. Activate NEXT MENU ITEM to step through the menu until **Save Memory to Disk = >NO<** is on the display. Activate CHANGE >ITEM< to start the saving process.
Use "**Load Memory from Disk = >NO<**" when you are loading a vocabulary into your DeltaTalker. Use "**Merge = >NO<**" when you are downloading a MAPMate.
10. On the computer, type in the name of the file you want to save the memory to. Press F1 to complete the transfer.
- 10a. To load memory, select the file on the computer that you want to load the memory from.

☞ If you experience a problem, you can change the MTI program's baud rate when you boot up the program. Type: dosmti<space>the new baud rate<enter>, For example, C:/dosmti9600<ENTER>



Tips for a Successful MTI+ Memory Transfer (Macintosh)



Plug in the battery charger before you begin a memory transfer.



DO NOT attempt a memory transfer by using the infrared capability. Use the cable that came with your MTI+ disk and connect your DeltaTalker and computer with the cable.

➤ The MTI+ does not work well on the following Macs running a system software version that is older than 7.5.3: Power Mac 6100, 7100 and 8100; Centris/Quadra 660av and 840av. You can update software from the following internet address: <http://asu.info.apple.com/>

➤ **The following is an especially important hint!**

If you are going to **download** vocabulary that was programmed for an 8 or 32 location overlay, make sure you are currently using the 8 or 32 set-up in the DeltaTalker. If you aren't, you must open the CHANGE OVERLAY menu and switch to the 8 or 32 keyboard and overlay **before** you download the vocabulary.

➤ If you are using a Macintosh that only has a **USB** port, you will have to purchase a "USB-to-Serial" adapter. PRC has tested an adapter made by *Keyspan*. Check the Keyspan web page or do a computer search for "USB to Serial Adapter".

➤ If you want to put the vocabulary on floppy disks, have at least 2 or 3 *formatted* disks ready **before** you begin the transfer.

➤ **AppleTalk** must be turned **Off** (inactive). **SerialKeys** must be turned **Off**.

➤ Once the transfer is in progress, do not plug anything into, or unplug anything from, the DeltaTalker.

➤ When a transfer is in progress, periodically check the computer screen and DeltaTalker display for any messages or problems.

➤ **If you experience a problem:** try changing the baud rate to 57600 in the MTI+ *Preferences* menu (inside the **Edit** menu). You may have to change the port setting. If you **are not** using a USB port, set the port setting to match the port your cable is in (i.e., Printer or Modem). If you are using a **USB** port, set the port recommended in the adapter instructions. The Keyspan adapter requires the **Printer** port.

➤ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the **Shift** key.

About Baud Rates

When the DeltaTalker and a computer are connected, they send information to each other through their serial ports. How fast the information travels from one serial port to the other is called the Baud Rate. The baud rate can be slow (300) or it can be very fast (57600). The important thing is that **the baud rate for your computer (or MTI program) and the baud rate for the DeltaTalker must be the same.**

The default baud rate for the DeltaTalker is 57600. You can change the MTI baud rate to match the DeltaTalker or you change the DeltaTalker's baud rate to match the computer's.

To Change the MTI Baud Rate to Match the DeltaTalker:

For the MTI+, click the **Edit** menu on the top of the computer display screen. Click **Preferences**. Click on the **Baud Rate** menu and select the rate you want.

To Change the DeltaTalker's Baud Rate to Match the MTI:

Turn the Toolbox On.

Activate the TRANSF. MEMORY menu.

Activate NEXT MENU ITEM to step to "MTI Baud Rate =>57600<.

Activate CHANGE >ITEM< until you step to the baud rate you want to use.

Activate EXIT.

Using the MTI+ for Macs

1. All memory transfers should be made with the **battery charger plugged in**. Plug the battery charger that came with your DeltaTalker into the CHARGER connector on the DeltaTalker. Plug the other end into a wall outlet.
2. All memory transfers should be made using the **MTI+ cable** that came with your MTI+, **not** the DeltaTalker's infrared capability. If you are using an older Mac, plug one end of the cable into the connector on the back of the computer that has the picture of the telephone handset or the printer above it. Plug the other end into RS232 connector on the DeltaTalker. If you are using a USB Mac, plug the cable into the USB port. Plug the adapter and other end of the cable into the RS232 connector on the DeltaTalker.
3. Turn the computer On and insert the MTI CD. Double-click on the icon to open the program.
4. **Copy the MTI+ to your Hard Drive for Better Performance:**
Click and hold on the folder labeled **MTI+**. Drag the folder to a location of your choice on your hard drive.
Close any open programs or applications. Then, double-click on the MTI+ folder on your hard drive to open the program.

Making a Transfer:

5. Open the EDIT menu at the top of your computer screen and click on *Preferences*.
6. Make sure DeltaTalker is selected in the **Device** menu.
7. Set the baud rate to the rate you want to use. The rate that is visible to begin with is the default baud rate for your selected device. This baud rate and the DeltaTalker baud rate must be the same.
8. Select a computer **port** settings. If you are using an older Mac, select the port you connected your cable to (either Printer or Modem). If you are using a USB Mac, the instructions that came with your adapter should tell you which port to select. The Keyspan adapter requires you to use the **Printer** port.
9. Do not change any of the **Miscellaneous** settings.
10. Open the DeltaTalker **Toolbox**.
11. Activate TRANSFER MEMORY MENU.
12. Activate NEXT MENU ITEM to step until **Save Memory to Disk = >NO<** is on the display. Activate CHANGE >ITEM< to start the saving procedure.

➡ Use the "Load" option when you are loading vocabulary **into** your device. Use "Merge" if you are adding new vocabulary to your current vocabulary.

13. Type in the name of the file you want to use.

➡ To **load** memory, go to the correct drive (or disk) and double click on the file name you want. Click on "Continue".

➡ **If you experience a problem:** try changing the baud rate for the computer to 57600 in the MTI+ *Baud Rate* menu under *Preferences* in the **Edit** menu on the menu bar. Remember, the baud rate for the DeltaTalker and the computer must be the same. Check that that you have selected the **correct computer port**. If you are using an older Mac, select the port that you plugged your cable into (Modem or Printer). If you are using a USB Mac, you must select the port that is indicated in your adapter instruction. The Keyspan adapter requires that you select the **Printer** port.

➡ **On older Macintoshes you may have to:** turn the computer OFF. Make your computer-to-device connections. Hold the computer **Shift** key down and turn the computer ON. Continue to hold down the **Shift** key until you see, *Welcome to Macintosh, All Extension Off*. Release the **Shift** key.

Transfer Vocabulary from One DeltaTalker to Another

➤ Read the "Tips for a Successful Memory Transfer" at the beginning of this chapter before you start a transfer.

By plugging the MTI+ cable in to the RS-232 ports on the back of each DeltaTalker, you can transfer *vocabulary* from one DeltaTalker to another. If you do not have the MTI+ cable, you can order it from PRC.

To Transfer Vocabulary from One DeltaTalker to Another:

1. Plug the MTI cable into the RS-232 connectors on the top of each DeltaTalker. (The ends of the cable are the same so it doesn't matter which end goes into which connector.)
2. Remove the keyguard and overlay from each DeltaTalker. Turn the Toolbox On in each DeltaTalker.
3. Activate TRANSFER MEMORY on the DeltaTalker that has the vocabulary you want to *transfer*. Activate NEXT MENU ITEM to step to Baud Rate. Make sure the baud rate is the same for both DeltaTalkers.
4. Activate NEXT MENU ITEM to step to: SAVE TO DISK = >NO<.
5. Activate CHANGE >ITEM< to begin the saving procedure. You see, "Looking for host computer...."
6. On the DeltaTalker you want to put the memory *into* activate NEXT MENU ITEM to step to Baud Rate. Make sure the baud rate is the same for both DeltaTalkers.
7. Activate NEXT MENU ITEM to step to LOAD MEMORY FROM DISK = >NO<.
8. Activate CHANGE >ITEM< to start the loading procedure.
9. The transfer takes place automatically.

Appendices

Appendix A: Computer Commands for Keyboard Emulation

Use the following commands when you are doing keyboard emulation for IBM Keyboards and Keypads

Create the <esc> command that you see below by activating CTRL[(Control left bracket) in your Toolbox. This sequence is shown as <esc> in all commands below. On your display you will see a small left arrow.

IBM Keyboard Keys

<esc>backspace.	<esc>f5.	<esc>pagedown.
<esc>capslock.	<esc>f6.	<esc>pageup.
<esc>delete.	<esc>f7.	<esc>pause.
<esc>down.	<esc>f8.	<esc>print.
<esc>end.	<esc>f9.	<esc>right.
<esc>enter.	<esc>f10.	<esc>scroll.
<esc>esc.	<esc>f11.	<esc>tab.
<esc>f1.	<esc>f12.	<esc>up.
<esc>f2.	<esc>home.	<esc>,hold,alt.
<esc>f3.	<esc>insert.	<esc>,hold,control.
<esc>f4.	<esc>left.	<esc>,hold,shift.

Specify a right or left key by placing and "r" or an "l" in front of the key name. For example,

To hold the *left shift key*: <esc>,hold,lshift.

IBM Keypad Keys

<esc>kp0.	<esc>kphome.
<esc>kp1.	<esc>kpins.
<esc>kp2.	<esc>kpleft.
<esc>kp3.	<esc>kpminus.
<esc>kp4.	<esc>kppgdn.
<esc>kp5.	<esc>kpplus.
<esc>kp6.	<esc>kpright.
<esc>kp7.	<esc>kpslash.
<esc>kp8.	<esc>kpstar.
<esc>kp9.	<esc>kpup.
<esc>kpdel.	<esc>kpnumlock.
<esc>kpdown.	<esc>sysreq.
<esc>kpend.	
<esc>kpenter.	

Computer Commands For Apple/Macintosh Computer Keyboards and Keypads

Create the <esc> command that you see below by activating CTRL[(Control left bracket) in your Toolbox. This sequence is shown as <esc> in all commands below. On your display you will see a small left arrow.

➤ If you are using an AlphaTalker, you can select <esc> from the MTI+ emulation menu.

Apple/Macintosh Keyboard Keys

<esc>backspace.	<esc>f9.	<esc>pause.
<esc>capslock.	<esc>f10.	<esc>print.
<esc>delete.	<esc>f11.	<esc>return.
<esc>end.	<esc>f12.	<esc>right.
<esc>esc.	<esc>f13.	<esc>scroll.
<esc>f1.	<esc>f14.	<esc>tab.
<esc>f2.	<esc>f15.	<esc>up.
<esc>f3.	<esc>help.	<esc>,hold,command.
<esc>f4.	<esc>home.	<esc>,hold,control.
<esc>f5.	<esc>insert.	<esc>,hold,option.
<esc>f6.	<esc>left.	<esc>,hold,shift.
<esc>f7.	<esc>pagedown.	<esc>,hold,alt.
<esc>f8.	<esc>pageup.	

Specify a right or left key by placing an "r" or an "l" in front of the key name. For example,

To hold the *left shift key*: <esc>,hold,lshift.

Apple/Macintosh Keypad Keys

<esc>kp0.	<esc>kpenter.	<esc>numlock.
<esc>kp1.	<esc>kpequal.	<esc>reset.
<esc>kp2.	<esc>kphome.	
<esc>kp3.	<esc>kpins.	
<esc>kp4.	<esc>kpleft.	
<esc>kp5.	<esc>kpminus.	
<esc>kp6.	<esc>kppd.	
<esc>kp7.	<esc>kppgdn.	
<esc>kp8.	<esc>kppgup.	
<esc>kp9.	<esc>kpplus.	
<esc>clear.	<esc>kpright.	
<esc>kpdel.	<esc>kpslash.	
<esc>kpdown.	<esc>kpstar.	
<esc>kpend.	<esc>kpup.	

Appendix B: Troubleshooting

➡ **You have checked your saved or loaded vocabulary and all the information does not seem to be there.**

Slow down the baud rate and try again. This is especially helpful for older computers.

➡ **You are trying to save a large file to a number of floppy disks and you get error messages.**

Save the file to the hard drive on your computer instead of floppies.

➡ **A transfer does not take place:**

Check the baud rates. The MTI and communication device baud rates must be the same.

If you are using a Macintosh computer: You may have to change the **port setting**. If you are **not** using a USB port, set the port setting to match the port your cable is in (Modem or Printer). The **Keyspan adapter** requires the port to be set to **Printer** to work.